USER MANUAL

4-wheeled walker with elbow support and brakes

Product code: JMC-C 3225

1. Characteristics

4-wheeled walker with elbow support and brakes is made of crosswise reinforced aluminium frame. The walker is equipped with solid 3" front swivel wheels, 5" rear wheels and hand brakes. Foam handgrips with length adjustment and elbow support provide proper stabilization and comfort of use. The height of the walker can be adjusted to the users height. When folded, the walker takes up little space, making it easy to store and transport.

Elements of the walker:

1. Elbow support
2. Handgrip with brake
3. Handgrip securing screw
4. Length adjustment
5. Folding system button
6. Height adjustment
7. Rear wheel 5"
8. Front wheel 3"

2. Application

4-wheeled walker with elbow support and brakes is an support equipment designed for people with reduced mobility who need stable support and assistance while walking. It is especially designed for patients who suffer from rheumatic changes, limited mobility in radiocarpal and elbow joints, and for patients with weakened grip strength. Recommended for the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the walker has a positive effect on the return to activity by people with lower limbs conditions and with locomotion or verticalization problems.

3. Assembly

UNFOLDING AND MOUNTING

Step 1. To unfold the walker, pull the sides apart to the maximum position until the handgrips are locked (characteristic “click” sound).

Step 2. Take out the upper frame part with elbow support and mount on top of the main frame, connecting it with locking pins.

Step 3. Install handgrips with brake into openings on the upper frame with elbow support and lock on desired length using locking pin. Secure them with screw (3) underneath elbow support (1).

Step 4. Mount legs with wheels onto lower part of the main frame and lock with pins (6). Make sure to have all legs on the same height.

FOLDING

To fold the walker, press the frame folding system button (5) located at the top of the main frame and fold the sides inwards.

HEIGHT ADJUSTMENT

- The height should be set on such a level that the user, who is leaning against the handgrips, stands upright with shoulders in rest position and with slightly bent elbows. It is inadvisable to hang on the walker with all the body weight.
- To adjust the height of legs, press the locking pin, select appropriate height and lock the position of all four legs. The legs are adjustable in 2.5 cm increments.

Note! The pins must be set on the same level in each leg.

LENGTH ADJUSTMENT OF THE HANDGRIPS

To adjust the distance between elbow support and handgrips, unscrew the securing screw (3) and press locking pin located underneath the elbow support (1). Adjust length according to user needs. Make sure length adjustment is identical on both sides before securing it with a screw.

4. Technical data

Overall width	55 cm
Overall length	54 cm
Height adjustment	117-132 cm
Length adjustment of handgrips	27,5-33,5 cm

Weight	5,00 kg
Maximum capacity	136 kg
Folded dimensions	69 x 16 x 110 cm

5. Notes

Timago International Group does not bear responsibility of improper use of the product, failure to comply safety regulations and misuse.

6. Maintenance

- Clean the frame of the walker with a damp cloth using water and mild detergent only. Dry thoroughly.
- **Do not use** for cleaning: strong polishing and abrasive agents, strong detergents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Remove dirt from moving components of the walker (hinges, clamping screws, locks) on a regular basis.

7. Storage and transport conditions

Product should not be exposed to direct sunlight, water or moisture.

8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

9. Medical device incidents

Any serious incident involving a medical device must be reported to the manufacturer and to the competent authority of the European Union member state where the user or patient resides.

10. Labels

	Reference number		Manufacturer
	LOT number		UDI code

	Serial number		Medical device
	Note		Please read the instruction
	The product meets the legal requirements of the European Union for medical devices (MDR 2017/745).		

Our complete line of products is available from our distributors. Learn more at: www.timago.com. Thank you for choosing Timago!

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