# USER MANUAL

## 4-wheeled walker with brakes and seat


Product code: DEFENDER

### 1. Characteristics

4-wheeled walker is made of crosswise reinforced aluminium frame. The walker is equipped with seat, backrest, brakes with parking function, front 8" swivel wheels with locking system and rear 8" wheels. Solid handgrips provide proper stabilization and comfort of use. The height of the walker can be adjusted to the user's needs. When folded, the walker takes up little space, making it easy to store and transport.

### Elements of the walker: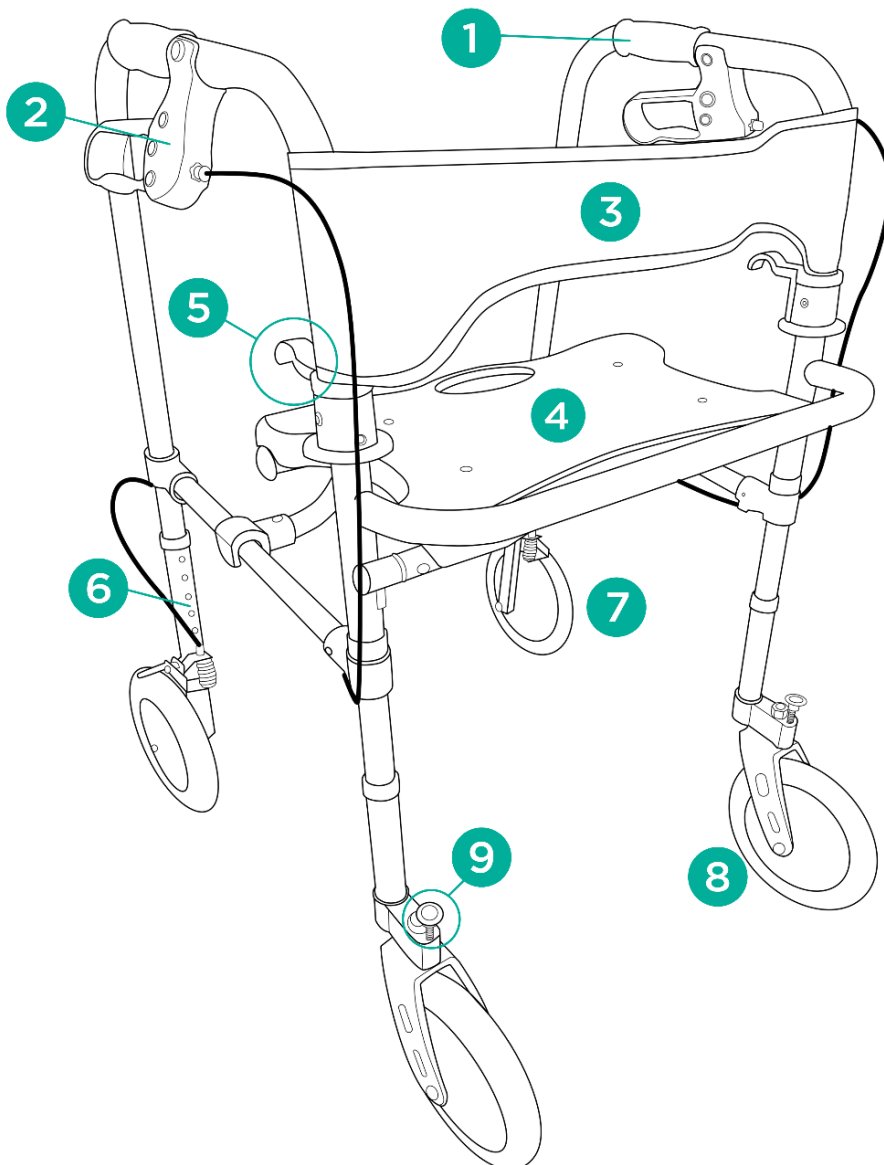

- 1 handgrip
- 2 brake with parking function
- 3 backrest
- 4 seat
- 5 frame folding system
- 6 height adjustment
- 7 rear wheel 8"
- 8 front wheel 8"
- 9 swivel wheel locking lever

## **2. Application**

4-wheeled walker is a support equipment designed for people with reduced mobility who need stable support and assistance while walking. It is also recommended for the elderly as an aid in keeping balance, improving walking comfort and ensuring safety from falling. The use of the walker has a positive effect on the return to activity by people with lower limbs conditions and with locomotion or verticalization problems.

## **3. Assembly**

### **UNFOLDING AND FOLDING**

- To unfold the walker, pull the sides apart to the maximum position until the handgrips are locked (characteristic “click” sound).
- To fold the walker, pull up the frame folding system levers located at the sides and fold them inwards.

### **HEIGHT ADJUSTMENT**

- The height should be set on such a level that the user, who is leaning against the handgrips, stands upright with shoulders in rest position and with slightly bent elbows. It is inadvisable to hang on the walker with all the body weight.
- To adjust the height of legs, press the metal pin, select appropriate height and lock the position of all four legs. The legs are adjustable in 2.5 cm increments.

**NOTE!** The pins must be set on the same level in each leg.

### **BRAKE FUNCTION**

- To stop the rollator, pull the brake handle up, then hold.
- When not in motion or using the seat, lock the parking brake by pulling the brake handle down.
- To release the parking brake, pull the handle upward.


### **SEAT FUNCTION**

- Walker is equipped with 48 cm wide seat allowing the user to rest.
- To put the seat up, grab the seat handle and pull it up.

## SWIVEL WHEELS LOCKING

- Front, swivel wheels can be locked to straight forward position.
- To lock the wheel pull the locking levers up and rotate them sideways (picture 1) and release the levers afterward. Make sure that both wheels are aligned in straight position. A “click” sound should be heard. Repeat on second wheel.
- To unlock swivels wheels, pull up and turn levers back to previous position (picture 1).

**NOTE!** Before using the walker make sure both wheels are either unlocked or locked.

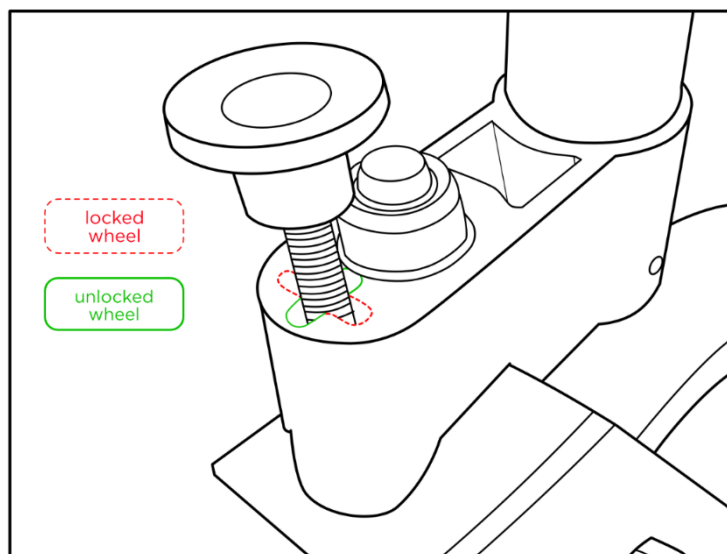Picture 1

## 4. Technical data

Overall width	67 cm
Inner width	49 cm
Overall depth	64 cm
Seat dimensions	48 x 29 cm
Height adjustment	90-101 cm
Weight	7,80 kg

Maximum capacity	136 kg
------------------	--------

## 5. Notes

**Timago International Group does not bear responsibility of improper use of the walker, failure to comply safety regulations and misuse.**

## 6. Maintenance

- Clean the frame of the walker with a damp cloth using water and mild detergent only. Dry thoroughly.
- Do not use for cleaning: strong polishing and abrasive agents, strong detergents, bleach, petrol or solvents (it may cause frame damage, visible defects and in consequence - loss of warranty rights).
- Remove dirt from moving components of the walker (hinges, clamping screws, locks) on a regular basis.

## 7. Storage and transport conditions

Device should not be exposed to direct sunlight, water or moisture.

## 8. Warranty information

All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained.

## 9. Labels

	Reference number		Manufacturer
	LOT number		Manufacture date

	Serial number		Medical device
	Note		Please read the instruction
	The manufacturer checked for compliance with essential medical devices requirements.		

Our complete line of products is available from our distributors.

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