iFold

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USER MANUAL

Electric wheelchair

Product code: iFold

1. Security information

- Read the manual before first use.
- Do not use the wheelchair until fully assembled.
- Use of the wheelchair by people who are unable to operate it independently is not recommended.
- Maximum load capacity must not exceed 150 kg.
- The wheelchair can only be used by one person at a time.
- It is recommended to keep your feet on the footrest and your forearms on the armrests when driving.
- Always check the battery level on the control panel before use.
- This electric wheelchair should only be used on a flat surface or at low elevations. It should not be ridden on a surface inclined at an angle of more than 6 degrees.
- Do not drive the wheelchair at maximum speed when going uphill or downhill.
- Do not use the wheelchair on soft and slippery surfaces such as mud, sand etc.
- Do not ride the wheelchair sideways on highly elevated areas.
- Do not put any additional load such as a trailer etc. on the wheelchair.
- Do not repair or modify the construction of the wheelchair by yourself. Use only original spare parts provided by the manufacturer.
- Do not stand up or sit down on the wheelchair when the control module is switched on.
- Do not use the wheelchair without anti-tip wheels.
- When the freewheel (manual) mode is on, do not get on or off the wheelchair without assistance.
- Do not lean forward to grab items laying in front, on the side or behind the wheelchair. Do not stand on the footrest. It might result in the wheelchair falling over.
- Avoid rapid changes of the direction while driving on elevations.
- Always slow down while driving downhill or making turns.
- Before use check if the wheels are correctly mounted.
- Do not make sudden moves while using the joystick.

- When storing, protect the wheelchair from rain and adverse weather conditions.
- Before turning on the wheelchair, make sure that the levers are set to electric mode.
- Avoid areas with horizontal gaps.
- If the wheelchair is not going to be used for a long period of time – remove the battery.
- Do not use the wheelchair on the road. It might result in a car accident.
- Do not store the wheelchair in places with high temperatures and humidity.

2. Product description and application

Electric wheelchair is designed for people who have problems with mobility, helping disabled patients with their everyday tasks. It can replace a standard wheelchair.

Electric wheelchair is suitable for patients with:

- disability, who have problems with unassisted movement,
- dysfunctions and reduced movement,
- heart and cardiovascular system diseases,
- weakening body due to advanced age.

A. ELEMENTS OF THE WHEELCHAIR

1. Headrest
2. Backrest with attendant handle
3. Seat belt
4. Mesh bag
5. Armrest
6. Armrest folding button
7. Control module with joystick
8. Flashlight/charger port
9. Seat (cushion)
10. Mudguard
11. Rear 12" pneumatic wheel
12. Front 8" solid wheel
13. Footrest

- 14. Folding mechanism engine
- 15. Back shock absorbers
- 16. Engines and operation mode levers
- 17. Anti-tip wheel
- 18. Front and folding mechanism shock absorbers
- 19. Lithium battery
- 20. Wireless remote control

B. PRODUCT CHARACTERISTICS

- Aluminium frame with electric folding
- Headrest included in set
- Pneumatic rear 12" and front solid 8" wheels
- Control module with LCD display and wireless control remote
- Safe and reliable electromagnetic brakes
- Fully closed, maintenance-free battery
- Security belt and anti-tip wheels
- Flip-up armrests and footrest
- Front, back and folding mechanism shock absorbers
- Mesh pockets for small goods located on the sides
- Pocket for goods located at the back
- Reflective stickers included
- Two modes of operation: electric and manual

C. TECHNICAL PARAMETERS

1. Product type: for indoor and outdoor use
2. Maximum speed: ≤ 6 [km/h]
3. Maximum capacity: 150 [kg]
4. Braking distance on horizontal surface: $\leq 1,5$ [m]
5. Max. range: 20 [km]

6. Battery: lithium, 24 [V] x 20 [Ah]
7. Max. angle of elevation: $\leq 6^\circ$
8. Max. turning radius: 1,2 [m]
9. Wheelchair weight: 31,50 kg (without battery)
10. Battery weight: 3,50 kg
11. Battery charging time: 6-8 hours

Note! Some parameters might vary depending on user weight, road conditions and battery usage.

- Safety Classification: type B
- Ingress protection level for engine: IP54
- Engine power level: 2x250[W]
- Wheelchair cannot be used in areas with flammable gas mixed with air, oxygen or nitrous oxide.

3. Icon description

Symbol	Meaning
	Safety Classification type B
	Important! Read the user manual
IP54	Protection against limited dust ingress and water spray from any direction.
	Fragile
	This side up
	Protect from humidity and moisture

4. Assembly and adjustment

- **Before unfolding, please read the manual, section 5A – Operation: CONTROL MODULE FUNCTIONS.**
- **The wheelchair cannot be unfolded manually.** Do not attempt to unfold the wheelchair manually, as this may cause damage to its parts. The warranty does not cover damage caused by forceful, manual opening or unfolding of the wheelchair.

- **Unfolding:** Remove the wheelchair from the box. Place it on the ground and attach the control module under the armrest using the screw (the screw is located under one of the armrests). Connect the control module cable to the wheelchair cable and turn on the module. Then turn on the wheelchair. Press the wheelchair function (7D) on the module twice. When the wheelchair enters P-2 mode on the display, press the volume -/+ button to unfold it.
- **Folding:** To fold the wheelchair, select P-2 mode on the control module and press the warning button (7H).
Note! Do not press the button to fold the wheelchair if it is still unfolding. Do not press the button to unfold the wheelchair while it is folding.
- **Control module adjustment:** Adjustments should be made while seated. Loosen the mounting screw located under the armrest, adjust the position or length of the module, and then tighten the screw afterwards.
Note! Before adjusting the module, ensure that the cable connected to it is unplugged
- **Armrest tilting:** Press the red button located on the outside of the wheelchair to lift the armrest. To lower the armrest, move it downwards till you hear click sound.
- **Anti-tip wheels assembly:** Insert the wheels into the mounting openings located at the back of the wheelchair. Pull the locking screw and push the wheels in until you hear a click sound.
- **Removing the battery:** To remove the battery, unplug the cable connected to it and press the switch located under the battery. Then remove the battery. To reinsert it, repeat these steps.
- **Headrest mounting:** To mount the headrest, remove the screws located above the attendant handle on the special metal plate. Place the headrest so that the holes on the plate align with the holes on the headrest frame. Secure the headrest using the screws and assembly key included in the set.
Note! The height of the headrest can be adjusted by pressing the button located behind it.

5. Operation

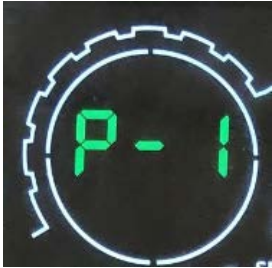
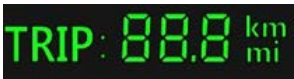
A. CONTROL MODULE FUNCTIONS

1. Battery level
2. Wheelchair functions indicator
3. Function menu/current speed display
4. Drive distance
5. Speed level indicator
6. Joystick
7. Operation buttons:
 - A. Volume -
 - B. Volume +
 - C. Turn on/off (unfolding)
 - D. Wheelchair functions
 - E. Speed adjustment
 - F. Light on/off
 - G. Horn
 - H. Folding

8. Light indicator
9. Joystick
10. Turn on/off button
11. Unfold (backrest reclining)
12. Operation buttons
 - A. Speed adjustment
 - B. Fold (backrest reclining)

Function	Displayed icon	Description
Battery level (1)		Shows current battery level. Green – high battery level, red – low battery level (charge immediately). When the battery indicator is flickering/flashng it means wheelchair requires battery charge.
Wheelchair function indicator (2), button (7D) and menu (3)	 	Displays the current function of the wheelchair. The green-highlighted part of the icon shows which part of the wheelchair is affected by the function: • When the backrest is highlighted – the backrest reclining function. • When the seat is highlighted – the wheelchair folding function. Press wheelchair function button (7D) to turn on the function menu on display (3). By pressing button again, you can pick up next option in menu (on default, P-1 is first option). To turn off the function menu, press button once again.
Speed display (3)		Display showing the current speed.
Drive distance (4)		Displays the driving distance.
Speed level indicator (5)		Indicates the current speed level. The lowest speed level is 1, the highest – 5.

Joystick (6 and 9)	-	Used to control the speed and direction. The further the joystick is pushed away from the middle, the faster the wheelchair will move. It will lose speed automatically after the joystick is released.
Turn on/off (7C)	-	Press to turn on or turn off wheelchair control module
Speed adjustment (7E and 12A)	-	Press to increase or decrease the speed.
Lights on/off (7F)		Press button to turn on/off the flashlight located under the control module panel. Note! Icon visible on LCD screen indicates that flashlight is on.
Horn (7G)	-	Press to use the horn (warning signal).
Backrest inclination buttons (11 and 12B)	-	The backrest inclination buttons (11 and 12B) are used to control the angle of the backrest with the remote control. To adjust the backrest, press and hold the folding button (12B). To reverse the angle of the backrest, press and hold the unfold button (11). The maximum backrest inclination angle is 55°.
Folding button (7H)		Used wheelchair folding function and reversing sound turn off/on (look: WHEELCHAIR FUNCTION MENU).
Reversing sound	-	To turn the reversing sound on or off (while the controller is turned on), press and hold the horn button (7G) and the warning button (7H) simultaneously for at least 3 seconds. When the

		number on display (3) changes to programming mode, release both buttons. Then, turn the controller off and on again. The reversing sound will now be disabled or enabled.
Wheelchair issue indicator		This icon appears when wheelchair malfunctions. To see possible causes check chapter 11 of the manual (Troubleshooting).
Bluetooth connection		Indicates connection to the remote control.

B. WHEELCHAIR FUNCTION MENU

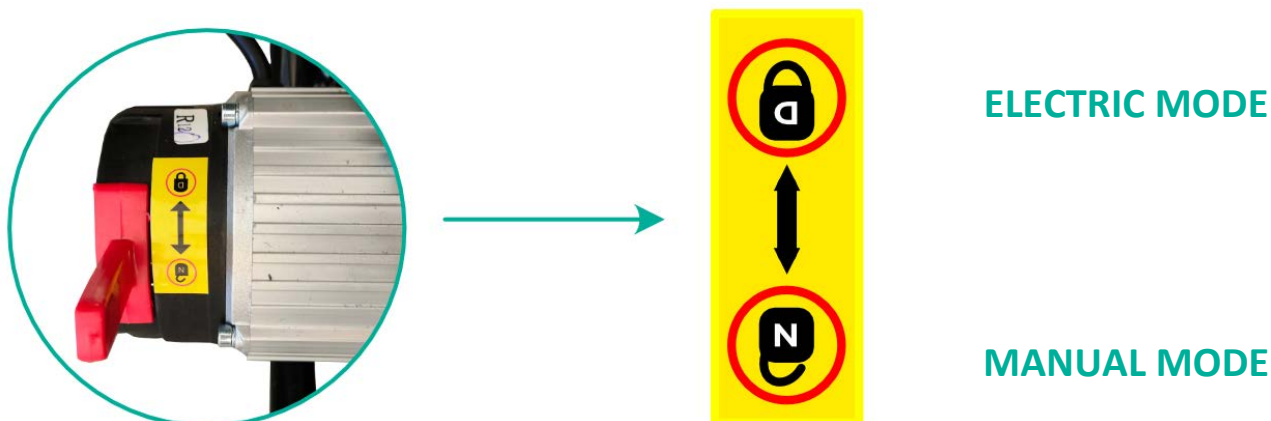
The wheelchair has a function menu that allows you to select one of two available functions: adjusting the backrest position or folding the wheelchair. To access the second function, press the button again. To exit the menu, press the button when the display shows the P-2 function.

Functions:

Function index	Function meaning	How to use function
P-1	Wheelchair backrest adjustment	When P-1 function indicator appears on the display, push joystick forward or backward to adjust the position of the backrest (angle of adjustment: 0-55°).
P-2	Wheelchair automatic folding/unfolding	When the P-2 function indicator appears on the display, press the warning button (7H) to start folding the wheelchair. To unfold it, press the speed adjustment button (7E) while P-2 function is on. Warning! Do not try to unfold the wheelchair while it is still folding!

C. MANUAL AND ELECTRIC MODE SWITCHING

To use the wheelchair in electric mode, move up the left and right engine levers located at the back. To use it in manual mode, move the levers down.

D. CHARGER

- Set includes a charger dedicated to the battery.
- Battery charging time is around 6-8 hours.
- Plug the charger into the charging port located in the control module (remove flashlight module before charging) or on the battery.

NOTE: Before plugging the charger in, turn off the control panel. If connecting charger to battery directly, unplug battery cables before charging.

- Red light indicates that the battery is being charged, while green light informs that it is fully charged. After the charging is finished, the charger turns off to avoid overcharging.
- Connect the charger to power supply.
- Check the connection.
- To avoid the risk of fire, do not modify or disconnect the charging circuit while charging the battery.

E. DIRECTIONS FOR USE

- Select the desired driving mode.

Note! Do not switch the mode while driving on elevation.

- Push the power button to check the proper functioning of brakes. Brakes are working correctly (electric mode) if you cannot push the wheelchair. Otherwise, contact the Seller.
- Tilt the footrest and sit comfortably on the seat. Lower the footrest back to horizontal position.

Note! Do not use the footrest as a support when sitting down or standing up from the wheelchair. This may cause the wheelchair to tilt and fall.

- Sit on the wheelchair and turn the control panel on. The LCD panel will light up. Joystick should be in neutral position.
- Take caution when using for the first time. Control the speed and direction carefully. The electromagnetic brake will get released only after pushing the joystick (electric mode). To speed up, push the joystick further away from its neutral position. To slow down – do the opposite.
- To stop the wheelchair, release the joystick. Pressing the power button will also stop the wheelchair.
- Using dedicated buttons on the control panel, adjust the wheelchair's speed according to user's physical state and road conditions.
- Wheelchair should only be used on flat surfaces. Driving on muddy or uneven surfaces can cause control system damage.

6. Technical data

Product code	iFold
Overall height	125 cm
Overall width	65 cm
Overall length	110 cm
Seat height from the ground	53 cm
Backrest height	58 cm
Seat cushion dimensions	49 x 46 x 6 cm
Headrest height from the seat	70-73 cm
Battery weight	3,50 kg
Maximum capacity	150 kg
Wheelchair weight	31,50 kg
Wheels	front: solid, swivel 8" back: pneumatic 12"

7. Wheelchair cleaning and maintenance

- It is recommended to regularly inspect the construction and its elements (wheels, frames, cables, control module, brakes etc.).

- Any used or damaged wheelchair parts need to be replaced (in such case, please contact the seller).

8. Battery maintenance:

- 1) Regularly check the battery level displayed. If the level is very low, charge it immediately to avoid a voltage drop (which may affect battery life).
- 2) If wheelchair won't be used for longer period of time, charge battery at least once a month.
- 3) Battery has a certain lifespan. Exchange your battery if it wears out.
- 4) Battery should be kept clean and away from kids. Avoid causing any shock or damage to the battery.
- 5) Charge it regularly to expand its lifespan.
- 6) Used batteries should be utilized according to local environmental regulations.

9. Tires maintenance:

It is recommended to inflate the tires to a maximum of 2.5 bar (35 PSI). The pressure should be adjusted according to the weight of the user and changes in air temperature. Tires may lose air over long periods of storage or non-use. Regularly check the tire pressure and their wear condition. When the tire tread pattern depth is reduced to 1 mm, replace the tires with new ones. Follow these instructions to inflate the tires:

a. Slight lack of air

- Press the tire evenly with your hands so that the tire and rim fit evenly.
- Inflate the wheel to the desired pressure.

b. No air

- Inflate the wheel to 30% of the total volume and press the tire evenly with your hands so that the tire and rim fit evenly.
- Inflate the wheel to the desired pressure value.

10. Storage and transport conditions

A. TRANSPORTATION

- Take caution during transportation. Do not throw, turn over or put pressure on the product and its packaging.
- For wheelchair folding, check „Assembly and adjustment”.

B. STORAGE

Wheelchair should be kept in dry and well-ventilated space. Avoid: high temperature and its rapid changes, acids, alkalis and corrosive chemicals.

C. STORING CONDITIONS

Temperature: -40[°C] ~+55[°C]

Humidity: ≤80%

Air pressure: 86 ~ 106 kPA

11. Troubleshooting

- In the event of a malfunction, the control module will indicate it with sound and warning appearing on the screen. The number displayed on the screen will help to identify the problem.

Issue indicator	Cause	Possible solution
Sound signal, display shows 0-5 number and issue symbol appears on the screen	Engine is set on manual mode / right engine failure	1. Turn off the control module, switch the engines from manual mode to electric mode, and turn off the controller again. 2. Disconnect the right brake connecting cable and connect it to the left brake connector. Do the same for the left brake connector. If the problem still appears, it is related to the controller. Contact the seller service. 3. Check the cable connections.
When pressing power button, controller/wheelchair is unresponsive	A. Battery problem B. Losely connected cabel or damage to the button C. Controller circuit failure	Check connection of the control module cabel. If cabel is connected correctly and not damaged, contact seller service for more information.
After turning on wheelchair, battery indicator on display shows red light + sound alarm starts.	Extremely low battery level	Charge battery for at least 30 minutes. Turn on wheelchair afterwards to see if problem still appears. If problem isn't solved, please contact seller service.
Sound signal, display shows 0-3 number and issue symbol appears on the screen	Left engine failure	1. Check if left engine is on electric mode. If not, turn off the control module, switch engine to electric mode and turn on module again.

		2. Disconnect the left brake connecting cable and connect it to the right brake connector. Do the same for the right brake connector. If the problem still appears, it is related to the controller. Contact the seller service. 3. Check connection of the cables.
After turning on wheelchair seven short alarm sounds can be heard	Joystick malfunction	1. Joystick is not in middle position. Turn off the module, bring back joystick to center position and turn on again. 2. Control module joystick is damaged, cable connected to the control module is damaged or it's not properly connected. Check connections of the cable. If alarm sounds appear again, please contact seller service.
After turning on wheelchair eight short alarm sounds can be heard	Internal malfunction	Contact seller service
Wheelchair doesn't stop after letting go of the joystick	Joystick malfunction	Contact seller service
Wheelchair does not move despite control module and joystick working properly without any alarm signals or indicators of error	Circuit related issue	Contact seller service
When pushing joystick forward, wheelchair turns left or right.	Issues with engine clutch	Contact seller service
When pushing joystick forwards,	Reversed wiring	Swap right and left brake connector cable with each other.

wheelchair backs up on very high speed		If problem still appears, contact seller service.
Wireless controller doesn't turn on Controller does turn on, but doesn't work	1. Batteries are depleted or they're not inside controller. 2. Controller issue	1. Put batteries included in set to controller or buy new set of batteries and switch them with used ones. 2. If controller turns on, but joystick or button doesn't work and control module doesn't show Bluetooth icon, contact seller service.

12. Warranty information

- All products distributed by our company are covered by a warranty, the terms of which are described in the warranty card available on our website. Please contact the retailer from whom you purchased the product. Please, keep in mind that for warranty purposes, the proof of purchase (receipt or invoice) should be retained. As a user-friendly company, we provide only tested products – in terms of material, quality and functionality. If you have any questions regarding service procedures, please contact us.
- Warranty does not cover:
 - wearable parts: upholstery, tires, armrests, battery
 - damages resulting from: self-repairs, changes in product construction, dropping, transportation, inappropriate usage, failure to follow instructions, natural disasters (fire, earthquake, flood).

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Zhejiang Hand In Hand Intelligent Health Technology Co., Ltd.
SUNGO Cert GmbH