REF EASY SHOWER

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USER MANUAL

KASUTUSJUHEND

KORSNIČKE UPUTE

NAUDOJIMO INSTRUKCIJA

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MANUAL DO UTILIZADOR

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NAVODILA ZA UPORABO

BRUKSANVISNING

PL INSTRUKCJA OBSŁUGI	5
CS NÁVOD K OBSLUZE	10
EN USER MANUAL	15
ET KASUTUSJUHEND	20
HR KORISNIČKE UPUTE	23
LT NAUDOJIMO INSTRUKCIJA	29
LV LIETOTĀJA INSTRUKCIJA	34
PT MANUAL DO UTILIZADOR	39
RO INSTRUCȚIUNI DE UTILIZARE	44
SK POUŽÍVATEĚSKÁ PRÍRUČKA	49
SL NAVODILA ZA UPORABO	54
SV BRUKSANVISNING	58

Basen do mycia głowy z prysznicem

Kod produktu: EASY SHOWER

1. Wstęp

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Product code: EASY SHOWER

1. Introduction

Before using the product check whether the equipment has no transport damage. If any damage is identified, please contact the retail outlet.

Then it is important to refer to the content of this manual that provides relevant information for the user.

If you have any questions concerning the use of the equipment, please contact us. Our personnel will provide you with the required information on an ongoing basis.

We would like to point out at the same time that if you have any questions not related to the product use, please contact your doctor, nurse or physical therapist.

2. Design

A portable, inflatable basin made of PVC for hair washing with a water drain outlet is soft and comfortable for use. Dual air chambers constitute a protection against spillage of water and suds.

The kit contains:

- Basin
- Water reservoir



- Shower
- Drain line
- Pump
- Reservoir bracket

3. Application

The hair washing kit allows the patient's head to be washed in the prone position. It is particularly recommended for indoor use during care for the bedridden and elderly people with difficulties in tilting their head and bending.




4. Method of use

- First inflate the basin using the pump and close the water drain valve at the bottom of the basin. The basin can be filled with warm water with a temperature below 50°C.
- Connect the water reservoir to the shower.
- Fill the reservoir with warm water with a temperature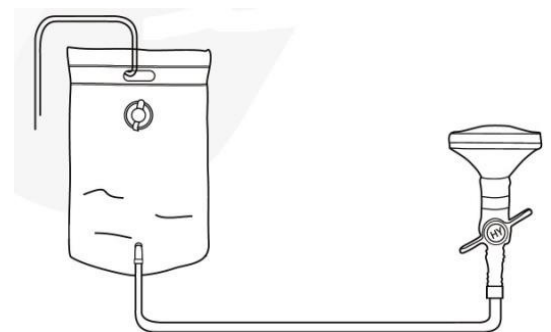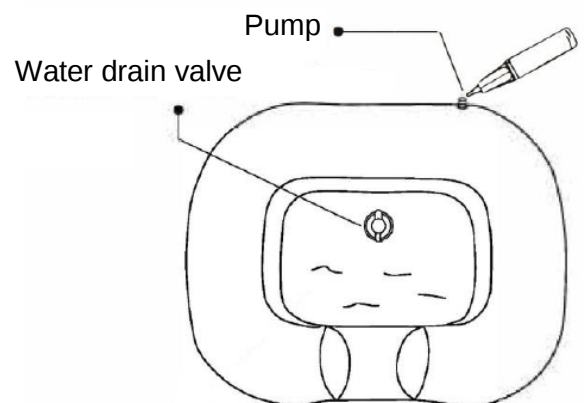

not exceeding 50°C. The reservoir can be hung on a stand or on the wall using the reservoir bracket.

- Prepare a bucket and place the drain line inside it. To ensure proper water drainage, check that the line is not kinked.
- Place the patient's head in the neck recess and unscrew the shower valve to adjust water flow.
- After washing the patient's head open the drain valve, so that the water can flow to the bucket through the drain line.

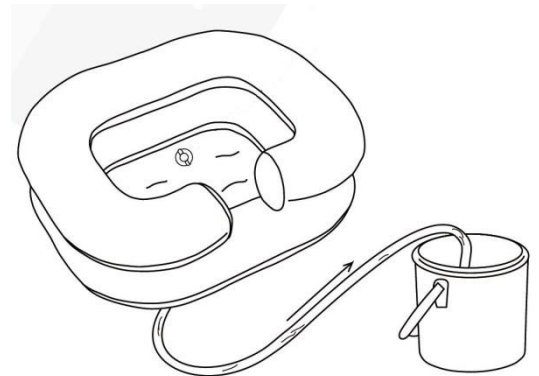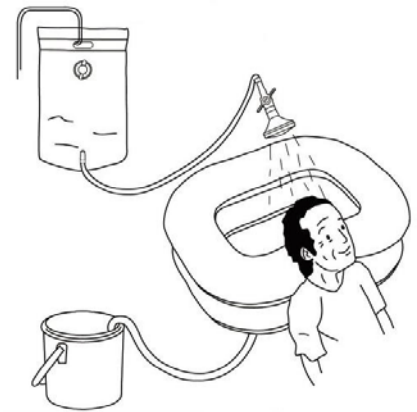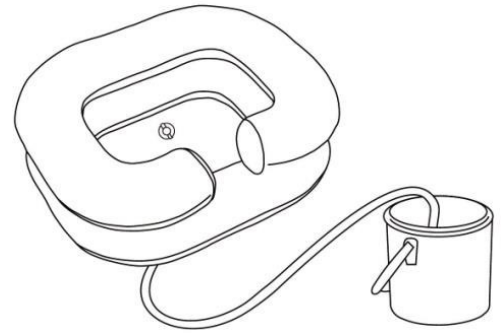5. Technical data

Basin dimensions	65 x 50 x 25 cm
Weight	800 g

Basin volume	10 l
Water reservoir volume	8 l

6. Maintenance and warnings

- Clean all parts of the kit after use and allow them to dry.
- Do not store the kit at a high temperature and in a humid environment.
- Do not expose the kit to direct contact with sharp objects.
- Adjust water temperature to avoid burns.
- Do not use detergents, solvents or essential oils.

7. Guarantee

All products distributed by our company are covered by a guarantee with terms and conditions set out in the guarantee certificate that you received when you purchased the product. We would like to point out that for the guarantee purpose you should retain the proof of purchase (receipt or invoice).

As a user-friendly company, we deliver only the products that have been verified in terms of materials, quality and proper functioning.

If you have any questions regarding the servicing procedure, do not hesitate to contact us.

8. Markings

	Reference number		Manufacturer
	Lot number		Date of manufacture
	Serial number		Medical device
	Attention		Refer to the manual
	The manufacturer assessed the product's conformity with the essential requirements for medical devices.		

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