Anytime!

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BRUGSANVISNING

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ZHEJIANG POCTECH CO., LTD.
Buildings 11 and 12 (North Zone of Floors 2 and 4), No. 1633, Hongfeng Road, Huzhou City, Zhejiang Province
Document No.: A4-CGM14-010
Version No.: V1.1

Prepared on: 20/09/2024


Timago International Group
Ossowski Sp. k.
Karpacka 24/12,
43-316 Bielsko-Biała, Poland

Anytime!

CT3 Series Quick Start Guide


Parts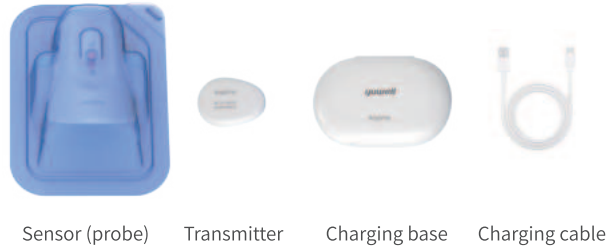

1 Preparations


Step 1 Download the software

- Turn on the mobile Bluetooth
- Use your smartphone' camera to scan the QR code on the package to download or update the App
- Follow the prompt to log in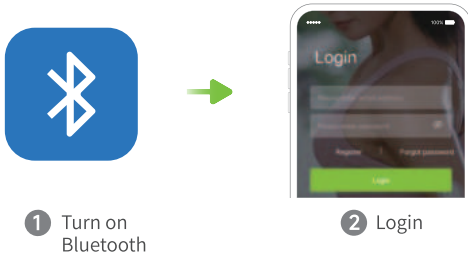

Step 2 Charge the transmitter


- Place the transmitter into the charging base for charging. The white indicator will flash (indicating charging).
- Wait for the indicator to turn blue (indicating full charging) and remove the transmitter.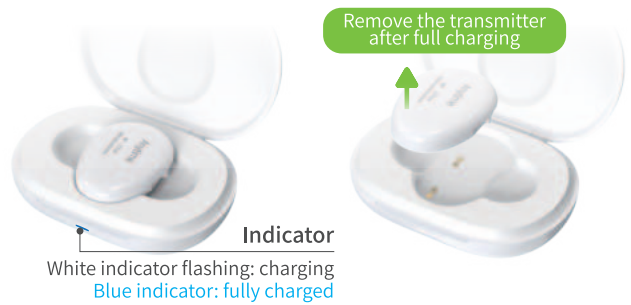

2 Connect the device


Step 1 Connect to the transmitter

- On the "Connect Transmitter" page of the App, select the SN corresponding to the transmitter to complete the connection.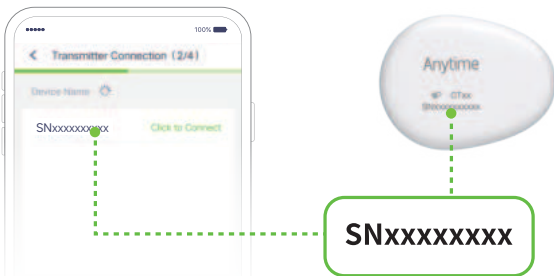

Step 2 Connect to the sensor

- On the App page, scan the QR code on the plastic housing of sensor and enter the initialization countdown mode.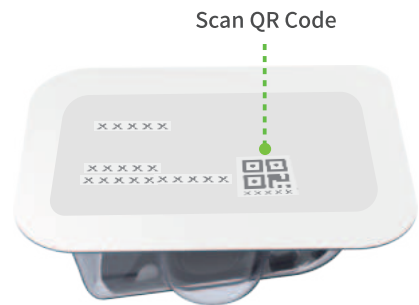


3 Implant the sensor

Step 1

Suggestion: wear on the abdomen or upper arm

- Choose an abdominal implantation (the site within 5 cm diagonally above the navel, in the dashed area shown in the figure), avoiding the belt, scars, and ribs.
- Disinfect with alcohol prep pad and air dry it before the next step
- To wear at the upper arm: please select the rear of upper arm side.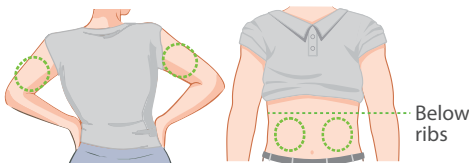


Step 2

Do not unlock before applying to the skin

- Tear off two protective films from the orange area.


Tear off protective films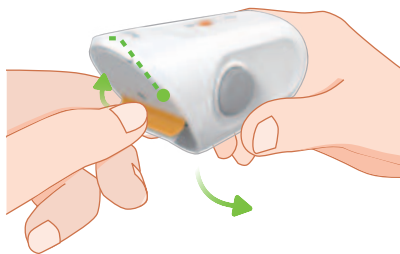

Step 3


Do not unlock before applying to the skin

- Place the sensor tightly against the implantation site.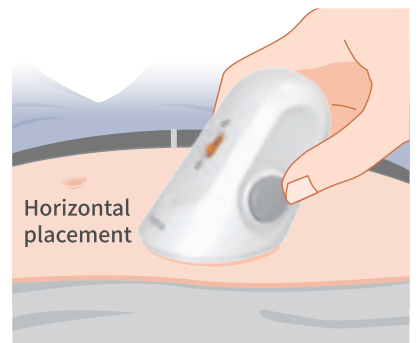


Step 4

- Slide the safety lock to unlock.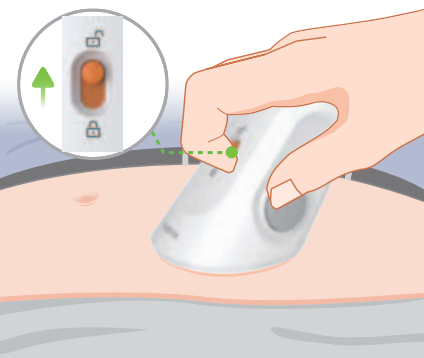


Step 5

- Press the buttons on two sides at the same time until a "click" sound is heard.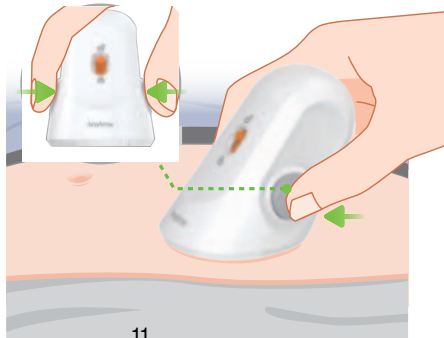

Step 6

- Slowly remove the insertion device part of the sensor to complete the implantation.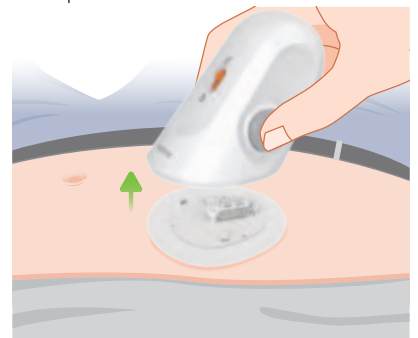

Step 1

★ Important steps

- Snap the transmitter onto the sensor according to the angle shown in the following figure.


Step 2



- Firmly pinch the transmitter and sensor as shown in the following figure until a "click" sound is heard.

⚠ Push the transmitter in reverse to verify it is locked.


1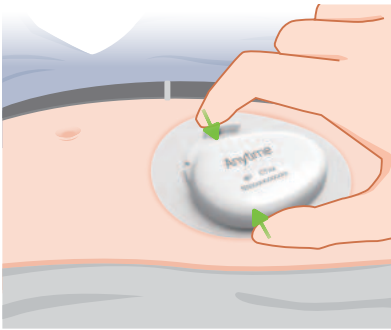


2

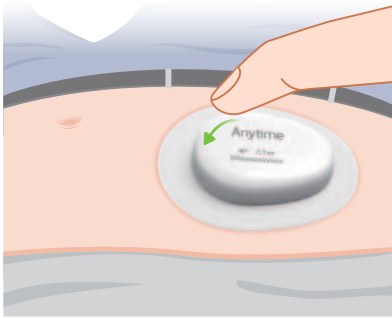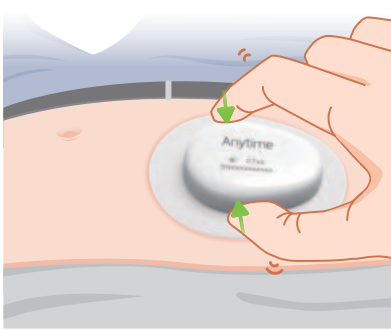- The App starts displaying the blood glucose value after initialization ends.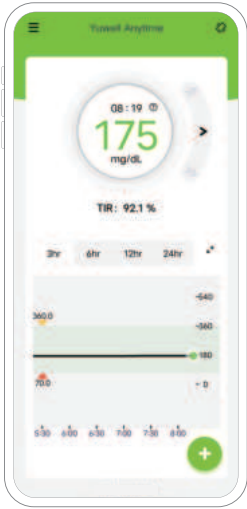

- Slowly remove as a whole with the tape and device from the wearing site.
- After removal, force apart the buckle downwards and slide the transmitter open.
- Dispose of the sensor as medical waste.

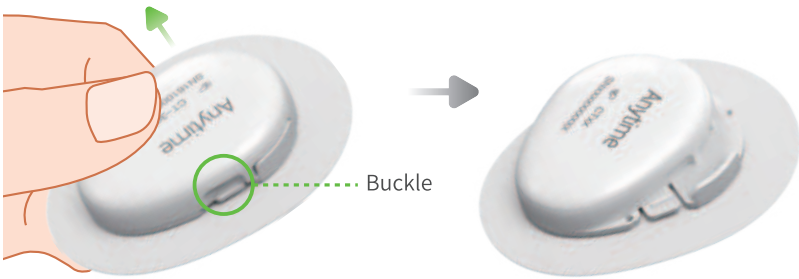⚠ Do not throw the transmitter by mistake

⚠ The transmitter is immediately charged

Anytime!

ZHEJIANG POCTECH CO., LTD.
Buildings 11 and 12 (North Zone of Floors 2 and 4), No.
1633, Hongfeng Road, Huzhou City, Zhejiang Province
Document No.: A4-CGM14-010
Version No.: V1.1

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