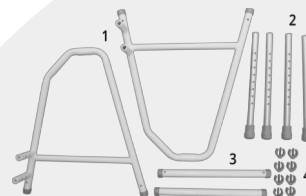
**Self-standing support rail**  
Item No. 302090

**Montage- und Gebrauchsanleitung**  
**Montage- og brugervejledning**  
**User Manual**  
**Manuale d'uso e manutenzione**

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## 1. Introduction

Thank you for buying a high-quality Mobilex product. This user manual contains a description of the medical device and important guidance to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

## Indications

The Self-standing support rail is intended for people who need support when sitting or who need to stand up when going to the toilet. It is made of aluminium tubing and is corrosion resistant. No tools are required for assembly.

## Contraindications

The Self-standing support rail is not suitable for patients who cannot maintain their body balance while sitting or standing. The freestanding and height-adjustable support must not be used if the user has perceptual problems. Contraindications cannot be related to a specific clinical picture and depend on the individual assessment of the potential user's state of health. The choice of the orthopaedic aid suitable for a particular clinical picture must always be made with the help of a specialist doctor or physiotherapist.

If you have any questions, please contact your dealer or Mobilex A/S.

<b>Mobilex A/S</b> Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 Email: info@mobilex.dk
--	--

## 2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Only use the product for the described purpose.
- The device must only be used on stable surface.
- When adjusting the device the user must take care not to squeeze the fingers or other body parts.

## 3. Description

### Product description

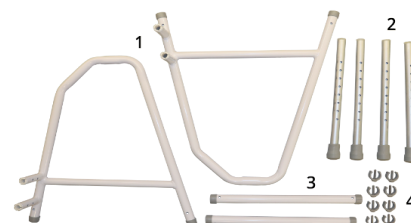
The Self-standing support rail is intended for people who need support when sitting or who need support when stand up when going to the toilet.

### Parts supplied and mounting instructions

The Self-standing support rail is delivered in a carton with the following contents:

#### Parts supplied:

- ✓ 2 side handles
- ✓ 4 legs
- ✓ 2 adjustable front bars
- ✓ 8 clips
- ✓ This instruction manual

### Assembly of the stool:

The Self-standing support rail is delivered in a cardboard box in which the lower parts of the feet are included separately. To assemble the Self-standing support rail ready for use, you must insert these 4 footrests into the side frame tubes provided and fasten them at the desired height using the enclosed clips, please make sure that the 4 legs is the same height (see illustration below). The two front bars can now be assembled using the enclosed clips. It is possible to adjust the width of the Self-standing support rail in three different widths with using the enclosed clips.

## Set Up description

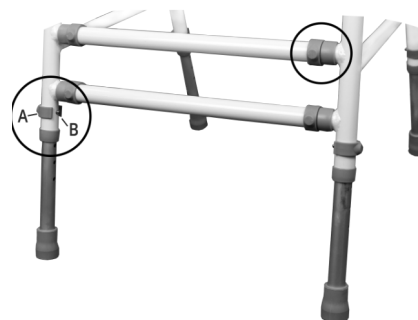
### Setting the desired height and width:

The Self-standing support rail is correctly set when the user can use the support and have both feet on the floor.

Grasp the **clip (A)** between your index finger and thumb and pull it out  
 Adjust the telescopic crossbars/legs to the desired width and height

Slide the **clip (A)** into the **hole (B)**.  
 Adjust the other telescopic tubes to the same height.

No adjustments should be made during use

**IMPORTANT!** Don't adjust the height and width during use. Always make sure all locking clips are mounted correctly.

## Accessories

No accessories for this product.

## Transport recommendations

The Self-standing support rail can be easily transported. Always assure that device during transportation is safely secured against unintended movements.

## 4. Technical Data:

Self-standing support rail	302090
Dimension (WxDxH) (mm)	468/493/518 x 532-615 x 616-786
Inner width (mm)	416/441/466
Outer width (mm)	468/493/518
Armrest height	616-786
Weight (Kg)	1.88
Max. Load (kg)	150

## Product label

The product label is located on main frame

Image of the product label shower stool 302090

	QR-code link to homepage		Manufacturing date
	Manufacturer		Max load
	Observe the user manual		Max width
	To be used indoor and outdoor		serial no
	CE marked product		Item Number
	Max load basket		Medical Device
	Only to be used indoor		

Overview icon meaning

## 5. Care & Maintenance

Normal dirt on metal and plastic parts can be removed with standard cleaning agents. Check the specific product information and use only commercial cleaning products which are suitable for cleaning and disinfection (no solvents or abrasives). Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional.

The device basically does not need maintenance. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to check the product at regular intervals by a professional service. We recommend the following to be checked:

- ✓ Cleaning and disinfection of the product
- ✓ Frame/product should be checked if it has been bent, damaged, worn or corroded
- ✓ Wear parts are checked and replaced if necessary

**Recommended service life:** Normally 10 years depending on use.

### Storage

We recommend storage in a dry place.

### Reuse

The product is suitable for reuse. Before reusing, where a new user is to take over the product, the product must be checked according to point 5.

## 6. Most common problems and solutions

Symptoms	possible cause	what to do?
The Self-standing support rail is not stable	Legs are adjusted to different heights	Adjust all legs to the same height

## 7. Disposal of the product

The device cannot be deposited with normal garbage. It must be delivered to the public station for re-use.

## 8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-marking and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly.

The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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(Konformitätserklärung / Konformitetserklæring / Dichiarazione di conformità)

**Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,**

**Hereby declare under our sole responsibility as a legal manufacturer** that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices.

The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

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### Harmonized norms used during conformity estimation:

**PN-EN 12182:2012, PN-EN ISO 14971:2012, PN-EN 1041:2009**

### ACCESSORIES LIST

Item nr.	Accessories item nr.
302090	No accesories available

### In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted

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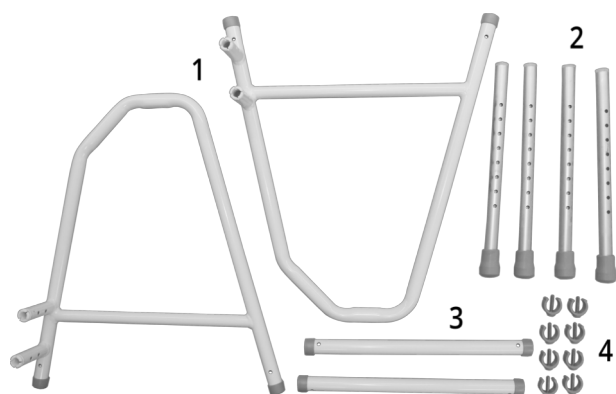
.....  
Signature/Unterschrift/Underskrift/firma

### Serial number / notes

**1** During the development of the armrests, the focus was placed on ergonomics to make it easier for user to stand up from the stool.

**2** The legs can be easily adjusted in height with clips and without tools. The pin of the clips is made of stainless steel.

**3** The width of the product can be easily adjusted with clips and without tools. The pin of the clips is made of stainless steel.

**4** The welded aluminium construction provides a lightweight and stable frame for safe use.