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PU foam and adjustable in depth

Item no.: 27840H, 27842H, 27844H, 27848H, 27851H, 27854H, 27857H

Two-layer foam. Top layer from viscoelastic foam

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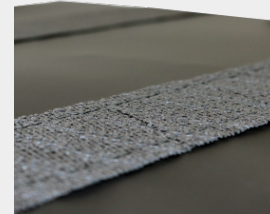
Standard wheelchair cushions
Made of good quality cold foam. High carrying capacity.
Available in the respective sizes with both PU foam and
viscoelastic foam with high density.
Polyester or incontinence cover (5 or 6 cm high)

Zipper on the back to easily remove the cover for washing. Rounded front to reduce pressure on the lower thigh.

Bottom with anti-slip material so it can be used on all wheelchairs regardless of the position of the velcro

All cushions are standard 5 cm high and available with 42-48 cm depth (adjustable). Available in 6 sizes (widths) from 40-50 cm Tested for 125kg.

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1. Introduction

This user manual contains a description of the product and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the product

The Wheelchair cushion is a medical device approved for 125 kg all with Incontinence cover, designed for making comfort when a person is sitting in a wheel chair and to reduce discomfort when sitting for a long time. The cushion is intended for use at home and in nursing or care institutions. The pillow is only intended for the purposes described above and any other use of the product is prohibited.

Indications and contraindications

The wheelchair cushions can be used in wheelchairs to achieve a good and pleasant comfort when sitting in the wheelchair. Contraindications cannot be linked to a specific medical condition, but starting to use the product and choosing the right version should be consulted with a specialist if necessary and safety recommendations should be followed.

By questions please take contact to your dealer or directly to Mobilex A/S

Mobilex A/S Grønlandsvej 5 DK-8660 Skanderborg Denmark	Phone: +45 87 93 22 20 Telefax: +45 87 93 17 77 E-Mail: info@mobilex.dk
--	---

2. Security instructions

- Make sure that this user manual is read by all persons using the device. Manufacturer doesn't take any responsibility for damages and/or injuries, caused by the fact that the user manual has not been followed.
- Only use the product in flawless condition.
- Follow the information on the product label.
- Only use the product for the described purpose.
- Always check that the label is readable and that the product is in good condition without signs of tear and wear such as loose seams, frays and material inconsistency.
- Do not leave the product unattended on the floor, a person may accidentally step on it with the risk of injury.
- Clean the product regularly.

3. Description

Product description

Standard wheelchair cushions

Wheelchair cushions are made of quality cold foam with high carrying capacity. The bottom of the cushion is made of anti-slip material, so it can be used on all wheelchairs, regardless of the position of the Velcro.

The front of the cushion is rounded to reduce pressure on the lower thigh. The cushions are equipped with a zipper on the back and 10 cm down both sides, so you can easily remove the cover for washing.

The cover is sewn from polyester or incontinence cover, which ensures great durability, as well as good stretchability.

The cushions are standard 5 cm high and available with 42-48 cm depth (adjustable) Available in 6 sizes (widths) from 40 to 50 cm.

The cushions are also available in the respective sizes with both PU foam and viscoelastic foam with high density. The standard cushions are available with black polyester or incontinence cover (5 or 6 cm height). All models are equipped with a zipper on the back and a non-slip bottom.

Adjustable universal wheelchair cushion

Universal wheelchair cushion in cold foam with black incontinence cover and adjustable seat depth. The cover is equipped with Velcro and can be easily opened. The cushion is made of cold foam and is equipped with grooves for depth adjustment of the cushion. The foam is cut to the desired depth. With Velcro the cover to the chosen depth. The cushion is supplied as standard with our wheelchair "Dolphin" and "Barracuda".

Cushions are available for seat widths from 40 to 51 cm. Seat depth can be reduced from 48 cm to 42 cm.

Seat cushion with 2 layers of foam (viscoelastic foam and cold foam)

The wheelchair cushion is constructed with a layer of viscoelastic foam on top and a regular cold foam on the bottom. Supplied complete with incontinence cover and non-slip underside. Available in 5 widths from 40 - 50 cm and in 2 depths.


Accessories

There are no accessories for the cushions.

4. Technical data

Item no.	Description	Material	Dimensions	Max. user weight
All	See page 16	See page 16	See page 16	125 kg

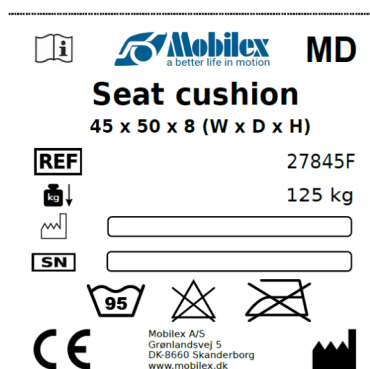Product label

	QR-code link to homepage		Manufacturing date
	Manufacturer	MD	Medical Device
	Observe the user manual	SN	serial no
	CE marked product	REF	Item Number
	Drip-dry		Medium Hot-iron
	washing temperatur		Do not tumble
	Handwash		Max width

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Oversigt icon betydning

5. Maintenance and care

The outer pillow cover is removable and machine washable in 95°C water. Please refer to the washing instructions on the label. The zipper of the pillowcase should be closed when washing. Do not use a tumble dryer to dry the pillowcase but air dry naturally.

The cushion itself cannot be soaked and washed in water, as this will affect its functioning. Any water that gets inside the foam will be trapped in it and will be very difficult to remove. Use only damp cloths to clean the pillow and dry naturally after cleaning. Do not wash, iron, or blow dry with hot air. This can damage the fabric of the product.

Recommended service life

Expected service life is 5 years; however, this will be subject to the actual conditions during use.

Storage and transport

The pillow allows for easy transport and storage. Store in a dry place. We recommend keeping the pillow packaging for ease of transport and storage. All cushions can ONLY be rolled and not folded or bent. Failure to do so may cause partial damage to the foam.

Reuse

The product is suitable for recycling. Before recycling, where a new user is to take over the product, the product must be checked according to point 5. After max. 2 years of use the product should be replaced depending on the conditions under which it has been used.

6. Trouble shooting

If the comfort of the cushion is reduced, please check if the washing or drying temperature has been too high or if the product has actually reached its lifetime. In doubt please contact your dealer who supplied the product. Always follow the instructions on the product label and check the product for deterioration.

7. Disposal of the product

The product may not be disposed of with the household waste but must be brought to the local recycling site.

8. Guarantee terms

Mobilex A/S offers 2 years of warranty for damages caused by errors in production or material errors. The warranty is not valid by not-intended repair or usage of the product. Parts subject to normal wear are not covered by the warranty, unless wear is caused by a manufacturing fault. If product changes are made without our written acceptance the CE-declaration and product warranty will not be valid. In the event of product damages covered by this warranty, please notify your dealer or Mobilex A/S directly. The warranty does not cover transport costs and does not include compensation for personal injuries or non-use of the product during repair. The warranty does not cover damages due to the user manual not being followed.

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Doc

Mobilex A/S, Grønlandsvej 5, DK-8660 Skanderborg, Denmark,

Hereby declare under our sole responsibility as a legal manufacturer that the product specified on the product list below, meet the essential health and safety requirements and is in conformance with the provisions of the Regulation (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on medical devices. The product specified on the product list below is "technical aid for the disabled", classified as Class I, medical device. The classification is based on the requirements of Rule 1 of annex VIII, of the Regulation (EU) 2017/745.

The CE marking has been affixed on the product according to Annex V of the Regulation (EU) 2017/745.

PRODUCT LIST

Wheelchair cushion with incontinence covers and friction bottom

PU foam Item no.: 278372, 278373, 27840N, 278402, 27840P, 27842N, 278422, 27842P, 27845N, 278452, 27845P, 27847N, 27847P, 27850N, 27850P

PU foam and adjustable in depth Item no.: 27840H, 27842H, 27844H, 27848H, 27851H, 27854H, 27857H

Two-layer foam. Top layer from viscoelastic foam Bottom layer from PU foam Item no.: 27840Q, 278407, 27840R, 27842Q, 278427, 27842R, 27842S, 27845Q, 278457, 27845R, 27845S, 27845F, 27847Q, 27847R, 27850Q, 27850R, 27855R

High density viscoelastic foam Item no.: 27840C, 27842C, 27845C

ACCESSORIES LIST

Harmonized norms used during conformity estimation:

PN-EN 12183:2014; PN-EN 12182:2012; ISO 7176-part 8, 16; PN-EN 14971:2012, PN-EN 1041:2009

EU2017/745 for Medical Equipment
(über Medizinprodukte / for medicinalprodukter / del Consiglio Europeo)

In case of unintended incidents or suspected faults, failures or defects of the product, the dealer or Mobilex A/S must be contacted /

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.....
Signature/Unterschrift/Underskrift/firma

Technical data:

Standard wheelchair cushions from PU foam

Tech

Item nr.	Description	Cover	W x D x H
278372	Wheelchair cushion from PU foam (H=50 mm) with friction bottom	Incontinence	37,5 x 40,0 x 5 cm
278373		Incontinence	37,5 x 43,0 x 5 cm
27840N		Incontinence	40,0 x 40,0 x 5 cm
278402		Incontinence	40,0 x 43,0 x 5 cm
27840P		Incontinence	40,0 x 45,0 x 5 cm
27842N		Incontinence	42,5 x 40,0 x 5 cm
278422		Incontinence	42,5 x 43,0 x 5 cm
27842P		Incontinence	42,5 x 45,0 x 5 cm
27845N		Incontinence	45,0 x 40,0 x 5 cm
278452		Incontinence	45,0 x 43,0 x 5 cm
27845P		Incontinence	45,0 x 45,0 x 5 cm
27847N		Incontinence	47,5 x 42,0 x 5 cm
27847P		Incontinence	47,5 x 48,0 x 5 cm
27850N		Incontinence	50,0 x 42,0 x 5 cm
27850P		Incontinence	50,0 x 48,0 x 5 cm
278552		Incontinence	55,0 x 48,0 x 5 cm
27840H		Universal wheelchair cushion from PU foam with friction bottom. Cushion is adjustable in depth	Incontinence
27842H	Incontinence		42,0 x 42-48 x 5 cm
27844H	Incontinence		44,0 x 42-48 x 5 cm
27848H	Incontinence		48,0 x 42-48 x 5 cm
27851H	Incontinence		51,0 x 42-48 x 5 cm
27854H	Incontinence		54,0 x 42-48 x 5 cm
27857H	Incontinence		57,0 x 42-48 x 5 cm

Two-layer standard wheelchair cushion (viscoelastic+PU)

Item nr.	Description	Cover	W x D x H
27840Q	Two-layer wheelchair cushion with one layer from viscoelastic foam (top) and one layer from PU foam (bottom) with friction bottom	Incontinence	40,0 x 40,0 x 8 cm
278407		Incontinence	40,0 x 43,0 x 8 cm
27840R		Incontinence	40,0 x 45,0 x 8 cm
27842Q		Incontinence	42,5 x 40,0 x 8 cm
278427		Incontinence	42,5 x 43,0 x 8 cm
27842R		Incontinence	42,5 x 45,0 x 8 cm
27842S		Incontinence	42,5 x 48,0 x 8 cm
27845Q		Incontinence	45,0 x 40,0 x 8 cm
278457		Incontinence	45,0 x 43,0 x 8 cm
27845R		Incontinence	45,0 x 45,0 x 8 cm
27845S		Incontinence	45,0 x 48,0 x 8 cm
27845F		Incontinence	45,0 x 50,0 x 8 cm
27847Q		Incontinence	47,5 x 42,0 x 9 cm
27847R		Incontinence	47,5 x 48,0 x 9 cm
27850Q		Incontinence	50,0 x 42,0 x 9 cm
27850R		Incontinence	50,0 x 48,0 x 9 cm
27855R		Incontinence	55,0 x 48,0 x 9 cm

Standard wheelchair cushion from high density viscoelastic foam

Item nr.	Description	Cover	W x D x H
27840C	Wheelchair cushion from high density viscoelastic foam with friction bottom	Incontinence	40,5 x 43 x 5 cm
27842C		Incontinence	42,5 x 43 x 5 cm
27845C		Incontinence	45,0 x 43 x 5 cm