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Introduction

We would like to thank you for choosing the MOBIAK S.A. electric wheelchair. The product User Manual is an important document that provides you information on the technical characteristics, operating modes, and after-sales support of the electric wheelchair.

Please read the user manual carefully before using the product. Read the "Safety" section carefully and use the product correctly according to the instructions in this manual.

- ✧ This manual contains operations, assembly methods, and simple faults solutions.
- ✧ This manual applies to our models: **0805720**.
- ✧ This manual contains wheelchair maintenance and self-checking methods, please put it in proper place.
- ✧ Please provide this manual for reference when other people are going to use this wheelchair.
- ✧ The annotations and illustrations in this manual might be slightly different with the real parts due to quality improvement or changing design. Please check it carefully.
- ✧ The wheelchair that you have in your hands may differ from the one in the following photos. The assembly process is similar.
- ✧ Contact with your dealer if there is any ambiguity or question.
- ✧ Improper use of any vehicle may lead to injury. Unsafe driving could harm yourselves and others.
- ✧ The electric wheelchair is intended to comfortably transport persons with walking difficulties or no walking abilities.
- ✧ This electric wheelchair is designed to transport 1 person only.
- ✧ In case of damage or malfunction of your product DOT NOT use the product and please contact the authorized and trained representative.
- ✧ Do not use the product for a purpose not indicated in this manual.
- ✧ MOBIAK S.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product.
- ✧ MOBIAK S.A reserves the right to change the information contained in this document without previous notice.
- ✧ There is a $\pm 3\%$. Deviation for the Dimensions of the Products.
- ✧ It is always recommended that you follow the procedures designed to deal with emergency situations.
- ✧ Always follow your doctor's instructions.

Warning sign.

Follow the instructions next to this symbol carefully.

Failure to pay attention and follow the instructions may result in physical injury or damage to the wheelchair or to the environment.

Product Performance

This series of electric wheelchairs are powered by battery, driven by motor. Users control direction and adjust speed by joystick controller. It is suitable for application of low speed, good road condition and small slope.

Main Structure

This series of electric wheelchairs are mainly composed of two parts:

- 1) Frame Part: frame, wheel, footrest, armrest, backrest etc.
- 2) Electrical Part: controller, motor, battery.

Intended Use

Our electric wheelchair is for disabled and elderly people who use it as a mobility vehicle, as well as for frail people who have difficulty on walking. It is suitable for outdoor use, for medium distances and is not suitable for use on motorways.

1. Safety instructions

Warning

Contraindications: The patients with mental problems, including the users who temporarily or permanently lose attention or judgment, are not allowed to use this product.

Attention

1.1 Electrical safety.

This Electric wheelchair has passed the test in accordance with ISO 7176-9 Climatic Test. This test can ensure that the users of the Electric wheelchair or attendants have sufficient time to move to safety place in a rainy day.

- DO NOT operate the Electric wheelchair under thunderstorm.
- DO NOT place this Electric wheelchair under rainstorm.

- DO NOT place this Electric wheelchair in a wet place for a long time.
- DO NOT spray, wash or wash the Electric wheelchair in an automatic car washer.
- Direct exposure to rain, smoke, or humid air may cause electrical and mechanical failure of the Electric wheelchair and may also cause rust.

Attention!

1.2 Conditions of use.

Ambient temperature: -10°C~50°C.

Relative humidity: 20%~90%.

Atmospheric pressure: 80kpa-123kpa; With no strong electromagnetic interference.

1.3 Driving on slopes.

- This Electric wheelchair has passed the test requirements of climbing on a slope of not exceeding 10° with a load of 120kg. DO NOT try to climb the slopes exceeding 10°
- During reversing on a slope, please be sure to slow down the Electric wheelchair. DO NOT reverse on a slope exceeding 10°, during reversing, please be extremely careful.
- The ability of climbing slope and driving distance are affected by battery and the current the conditions of use. For example: The weight of the users, the land morphology (such as grass or gravel), Gradient of hill, Battery life and service life, extreme temperature, Use and weight of the attached accessories, etc.
- Avoid driving on steep slopes. The slope range should be up to 10 degrees.
- Keep moving forward when going uphill or downhill, maintain a low speed when going downhill.
- DO NOT drive on steps and avoid pit or potholes.

1.4 Braking

When the Electric wheelchair brakes, please sit properly, steadily hold the handrail and lose controller joystick, the Electric wheelchair will stop in several seconds.

Note: Electromagnetic brake will not take effect immediately, it will take effect after the wheel rotates for 1/2 cycle.

1.5 Emergency Braking.

In case of critical situation or unexpected movement, press the power switch key to make the Electric wheelchair stop (Although this emergency brake is effective, it must not be used under normal conditions).

Note: Frequent use of emergency brake may lead to the motor fault, which further causes that the Electric wheelchair cannot be operate normally.

1.6 Use of mobile phone

During operating the Electric wheelchair, DO NOT use mobile phones or wireless devices.

Use of mobile phones or wireless devices will generate strong electromagnetic fields, the electronic system of the Electric wheelchair will be interfered. If the mobile phones or wireless devices have to be used, the Electric wheelchair has to be stopped and powered off in advance. Follow the Highway Code.

1.7 Protection from Electromagnetic Interference

The wheelchair should away from strong magnetic fields and large inductive electrical equipment. Pay attention to that if there are sources of electromagnetic interference nearby, as far as away from those sources to avoid electromagnetic interference.

Warning!

1.8 General safety warnings (read carefully)

- Please practice in parks or other safe open places until you can use the wheelchair skillfully.
- Fully practice driving in a safe place, to master principle of moving forward, stopping and turning circles.
- Please strictly follow traffic rules of the pedestrian, do not regard yourself as vehicle driver.
- Please drive on the sidewalk and the zebra crossing. Do not drive on vehicle lane.
- Steadily drive, avoid driving in 'Z' line or sharp turn.
- Please keep pressure of pneumatic tire properly. The abnormal tire pressure may cause unsteady driving or excessive loss of battery energy.
- DO NOT try to pick up objects by bending down when sitting on the Electric wheelchair, the Electric wheelchair or user may roll over or be damaged.
- DO NOT use the Electric wheelchair on escalators, which may increase the risk of being injured.
- DO NOT drive in a vehicle lane or anywhere other than on pavements.
- DO NOT drive on the surface of water, oil, ice or other slippery objects. In case of violation, the Electric wheelchair may be out of control.
- DO NOT attempt to cross over obstacles and ditches exceeding 45mm, violation of this operation may cause personal injury.
- DO NOT turn sharply and do not reverse at high speed.
- DO NOT carry passengers.
- DO NOT try to drag another Electric wheelchair.
- DO NOT operate the Electric wheelchair without checking the fasteners, connectors or detachable parts for correct connection and fastening. Please always pay attention to the surrounding environment.
- DO NOT repair the parts, accessories, or adapters of the Electric wheelchair without authorization.
- DO NOT connect other medical equipment (such as life support equipment and respirator, etc.) to the electronic system of the Electric wheelchair.
- DO NOT operate the Electric wheelchair when your judgement may be affected (such as drink, take drugs or take medicine);
- DO NOT operate the Electric wheelchair if you suddenly DO NOT feel well.
- DO NOT operate the Electric wheelchair when your vision is severely weakened.

- For the injury accident caused by violation of local or international laws or regulations, we undertake no responsibility.
- All the functions can only be operated after you correctly sit on the seat.
- DO NOT place your feet in any part out of the footrest.
- DO NOT stand on the seat of the Electric wheelchair.
- DO NOT try to drive on a slope exceeding 10°.
- DO NOT reverse on a slope exceeding 10°.
- When driving on a slope, if you release the joystick of the controller, the Electric wheelchair may move back. When moving forward or backward, the Electric wheelchair will roll back about 30 cm before the brake function takes effect.
- Before use of the Electric wheelchair, please check the connection of all electric parts for safety and stability.
- DO NOT disconnect, snip or modify the wiring harness ports which have been installed on the Electric wheelchair or connected to the Electric wheelchair under any condition.
- DO NOT use other unqualified batteries, for example, the lead acid batteries which cannot be recycled, etc.
- Before installation, please read the information related to batteries and battery charging.
- If you don't read this user manual, it is preferable not to use this product or another available part.
- If you don't understand the usage or the precautions, please contact with the dealer or the proper technical person before you use the product because may be caused damage.
- Be careful when children are nearby and do not allow the children to play with the product.
- Do not exceed the maximum payload.
- Special care is always required where there are moving parts that can cause limb entrapment and injury.
- Do not attempt to lift the wheelchair by any removable parts.

In case of non-Compliance of the above requirements, there may be unexpected injuries or damages.




2. Product Introduction

2.1 Product features and functions

- The DC brush motor which is the core part to drive the Electric wheelchair. The motor is featured for its large torque, high efficiency and power saving.
- The DC brush motor has the advantages of high performance, long service life and low power consumption. The noise of the motor can be controlled within 65dB.
- Using lithium battery has high density energy & long life.
- The intelligent control of motor can be done by using the controller with the joystick. Can be done the following actions: forward movement, differential turning and low speed reversing, which makes the control of the product easier.

- The frame structure uses foldable four-bar linkage mechanism achieves quick and efficient folding and unfolding actions. The Electric wheelchair is equipped with the function of electric folding or manual folding, and it has the advantages of being safe, reliable and simple to operate.
- The foldable frame structure has the advantages of portability. It is compact and easy to carry (trunk and air transportation) since it does not occupy too much space.
- A gyroscope sensor detects the posture, and an audible warning is activated when there are potholes and big ground slope. When the speed is lowered to gear 1, will pop up on the screen and the relevant warning.
- In case of no operation (with a key, the joystick, or the APP) within 30 minutes, the Electric wheelchair will power off automatically.
- To enter the cruise control mode, keep a long press of "speed +". Then push the joystick forward so the Electric wheelchair start moving and to adjust the speed.
- Wireless controller. The wireless controller may achieve the front, rear, left and right movement of the Electric wheelchair. Also, the key to control the electric folding and unfolding can be used.
- There is a mode for novice users.

2.2 Component introduction & Structure of the product (unit: mm)

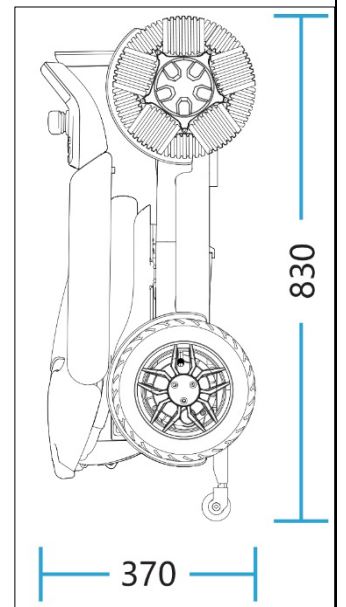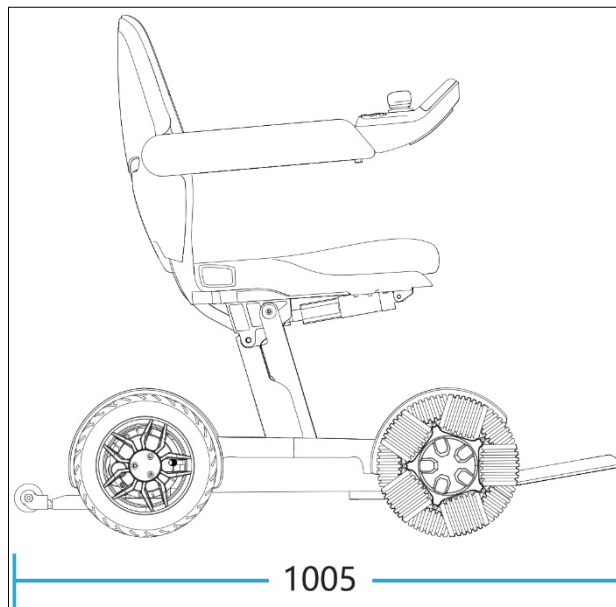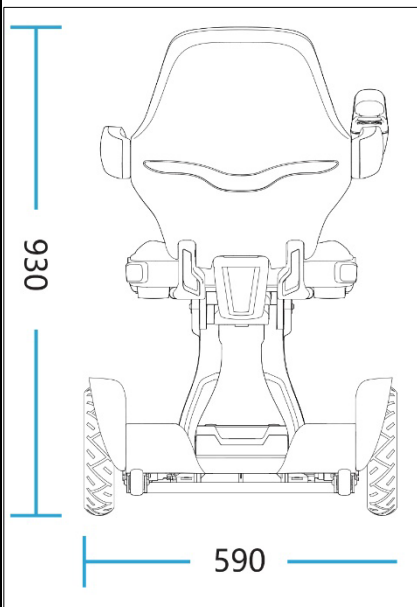2.3 Technical Characteristics of the product 0805720

Folding mode	Electric	Height of handle	280±5 mm
Motor parameters	DC motor, rated voltage 24V, Rated power 200W*2	Foot plate	Fixed on the frame & foldable
Lithium battery parameter	1 pc X 24 V DC, 20 AH	Suspensions	Yes, Front
Battery weight	1 pc X 3,4±0,15 kg	Ditch width	100 mm
Total weight	42,1±2 kg	Max User Weight	120 kg
Speed range	≤ 6 km/h	Minimum radius of gyration	760 mm
Autonomy- Travel Range	25 km	Static stability	20° for longitudinal direction, and 15° for side direction
Max Slope	10°	Dynamic stability	10° for upslope, 10° for downslope
Front and rear wheel specifications	The front wheels are 10" omnidirectional wheel, and the rear wheels are 10" pneumatic with PU tire (2.4MPa-2.7MPa)	Brake engages on level road	100 cm
Brake	Electromagnetic brake	Overall dimensions	L:1.075 mm, W:628 mm, H:930 mm
Charger parameters	The entered voltage is 29,4V and the output current is 4A	Folding Dimensions	L:830 mm, W:628 mm, H:370 mm
Maximum controller output power	30 A	Anti-tip device tipping angles	18°
Seat	Fabric	Angle between leg and seat surface	0-10°
Seat Wide	420±5 mm	Horizontal shift of the shaft	81±5 mm
Backrest height	450±5 mm	Vertical shift of the shaft	528±5 mm
Plane angle of the seat	-7±0,5°	Pivot width	1180 mm
Backrest angle	21°±1°	Reversing width	1180 mm
Effective seat depth	420±5 mm	Ground clearance	64 mm
Distance from the footplate to the seat	423+5 mm	Required width of angled corridor	1000 mm
Surface height of the front seat	510±5 mm	Required doorway entry depth	1070 mm
Armrestes	Foldable	Required corridor width for side opening	1150 mm
Other	Mud-fender, Safety Belt, Anti-Tip castors.		

2.4 Package Contents

The package contains, the Main unit x1, the Battery x1, User's manual the x1 and the Charger with the power Cable x1.

2.5 Storage conditions

- The Electric wheelchair should be stored in its packaged, at temperature of $-40^{\circ}\text{C} \sim 60^{\circ}\text{C}$.
- The humidity has to be no more than 95%
- The Atmospheric pressure shall be between 56~146 kPa.
- The Electric wheelchair shall be stored in a room without the harmful gas which may cause corrosion and with good ventilation. It is also not allowed to be stacked and pressurized.
- When packed, the maximum stack height is three layers.
- DO not store the product for a long term into places which get close to high temperature source or direct sunlight.
- If you need to store it for a long time, fist fully charge the battery and then turn off the power switch.
- If the product has been stored for more than a month, fully charge the battery and then use it or continue storage. That is to check the level and health of the battery every month.
- After long-term storage, all batteries in the first charge cycle provide less capacity than rated capacity. Full capacity can be reached after several charge/discharge cycles, usually at least 3 cycles.

3. Use of Functions

Attention!

3.1 Driving Notice

- When operating the Electric wheelchair for the first time, try to choose a flat and hard ground and ensure that there are no obstacles and pedestrians around you.
- The maximum load is 120 kg. DO not drive overload. Limited use for 1 person.
- Ensure that the Electric wheelchair is completed unfolded.
- Ensure that you have fastened the safety belt before you start moving.
- Sit up properly on the Electric wheelchair and gently push the joystick in the direction you want. With

this way it moves forwards or backwards and turns left or right. After release the joystick it will stop. Practice these basic functions until you are familiar with the handling of the wheelchair.

- When driving, make sure there are no obstacles around you.
- When driving. DO NOT steer on pedestrians.
- When operating the electric wheelchair in relatively difficult conditions (such as entering the door or steering), make delicate handling. In difficult conditions, it is recommended to select the lower speed.
- Check that the condition of the electric wheelchair is good and that it works properly.
- The electric wheelchair will switch off automatically after 30 minutes of inactivity.

3.2 Folding and Unfolding


In the process of folding or unfolding, if you swing the joystick, this operation will stop. After the intermediate stop, you can continue this operation.

Folding

Press "horn key" and "speed -" at the same time for 3 seconds

Unfolding


Press "horn key" and "speed +" at the same time for 3 seconds.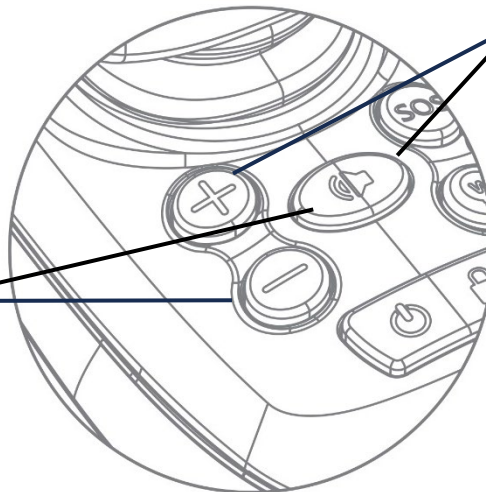

The Folding and Unfolding Operation, if there is no other command, will stop automatically after finishing.

3.3 Controller with Joystick

Controller Introduction: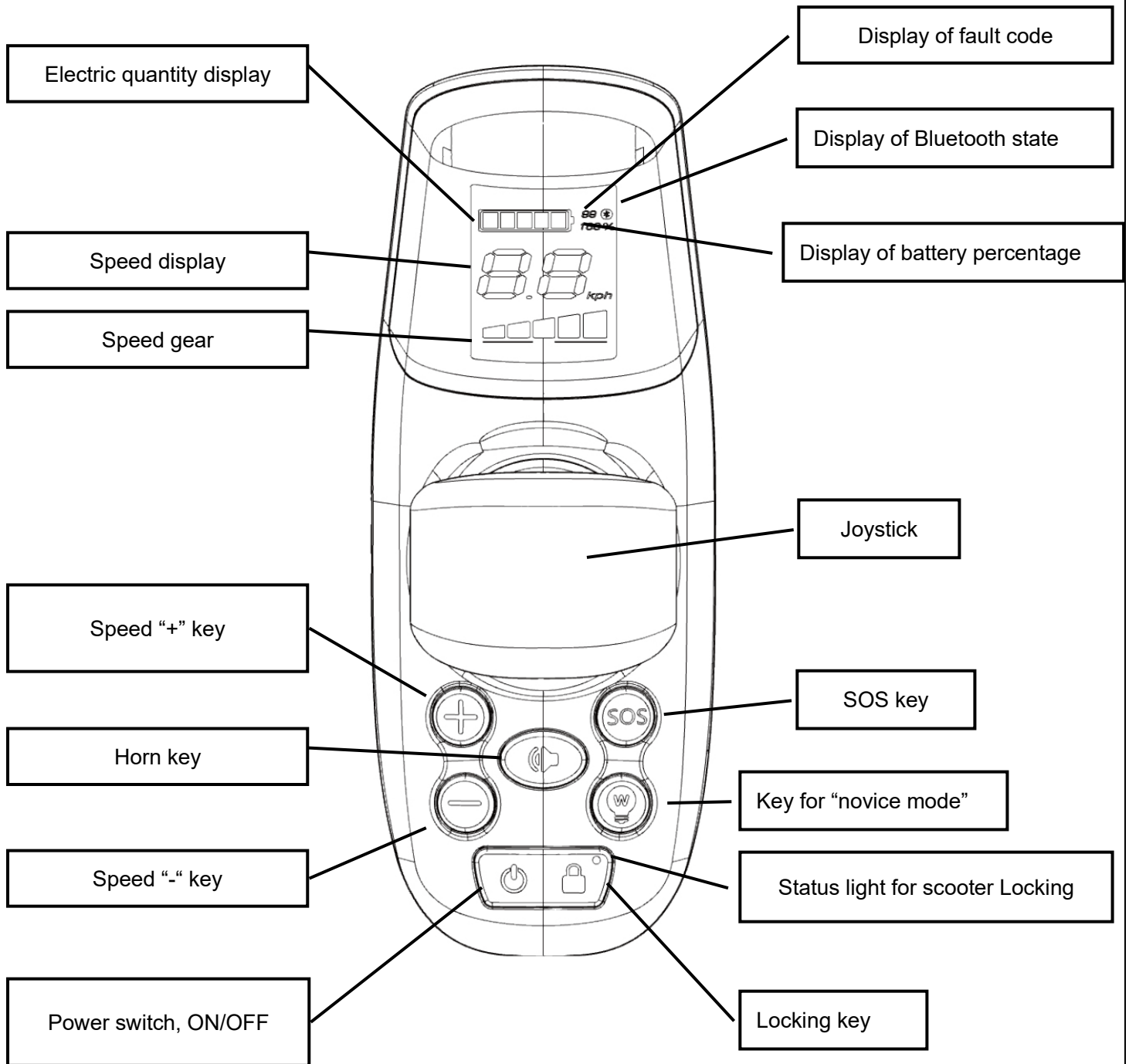

Controller Instructions:

Joystick

By pushing the controller Joystick, the direction and speed of the Electric wheelchair can be controlled.

Power switch & locking key.

Long press for 2 seconds to turn on the power.

Long press for 2 seconds to turn off the power.

Press the power key, the locking indicator of the Electric wheelchair is off, the Electric wheelchair is unlocked.

Press the power key, the locking indicator of the Electric wheelchair is on, the Electric wheelchair is locked.

Speed “+” key.

When it is pressed, a higher speed is selected.

Keep a long press of "+" key to enter cruise control mode and push the joystick forwards to start.

Speed “-” key.

When it is pressed, a lower speed is selected.

Horn key.

Keep a short press of the hone key, the horn will ring for once.

Locking Status light for Electric wheelchair.

After powering on, the locking function will be entered in default; During use, locking mode can be entered under any control mode.

Under locking function, the joystick cannot control the motor to move.

After unlocking, the motor can be controller to move.

Under locking mode, after the APP is connected, there will be a pop-up box to remind locking mode.

SOS key.

Press the button for 3 seconds. The system activates the audible signal.

Key for “novice mode”

The driving parameters of the Electric wheelchair are automatically lower adjusted to gear and data suitable for novices to practice.

LCD screen menu Instructions:

Electric quantity display

The device controls the electric quantity display light, displays the electric quantity in real time, and displays it in 5 grids.

Speed display.

LCD screen displays the driving speed of the Electric wheelchair in km/h.

The precision of the displayed speed is $\pm 5\%$.

Speed gear

Press “+” key for once, the speed will be increased to the next gear, gear 5 is the highest. Press “-“ key for once, the speed will be decreased to the next gear, gear 1 is the lowest. The above function can be also achieved through APP.

Display of fault code

For the content of the codes displayed by the product's built-in diagnostic test system, see the section, "Troubleshooting".

Bluetooth status display

When Bluetooth is not connected, the Bluetooth indicator on the LCD screen is off.

When Bluetooth is connected, the Bluetooth indicator on the LCD screen is on.

Indication of the percentage of remaining electrical power



The Remaining electric quantity of the battery is displayed in percentage.

The above function can also be achieved through APP.

3.4 Battery



Battery Removal:

First open the battery lock, located on the frame. Then, take out the battery upwards.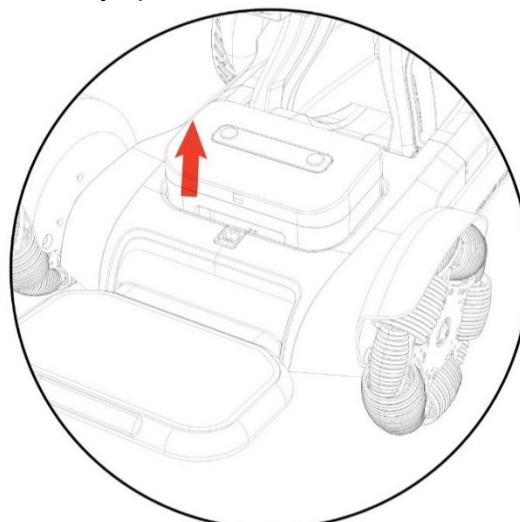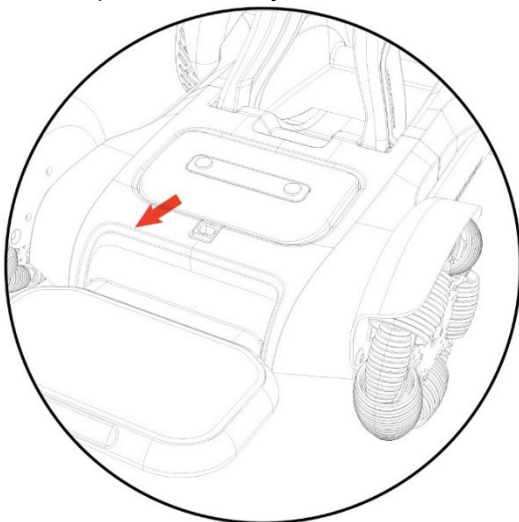


Battery Installation:

First, insert the battery into the battery slot and then close the lock.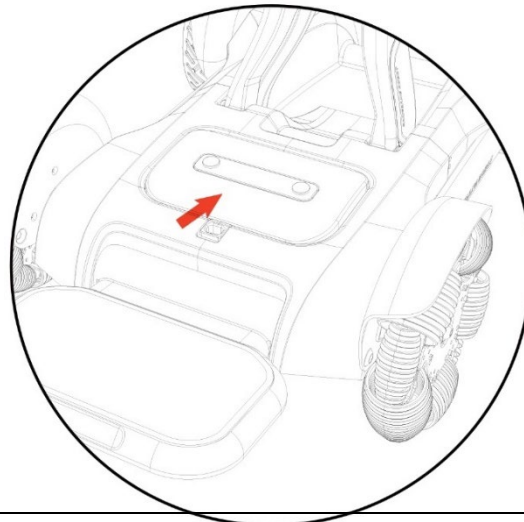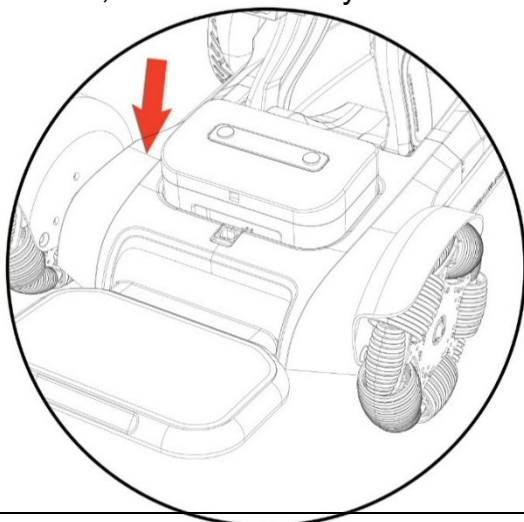

3.5 Switch to pushing mode.

Press “+” key and “-” key at the same time to enter pushing mode. You can quit from this mode by wing the joystick.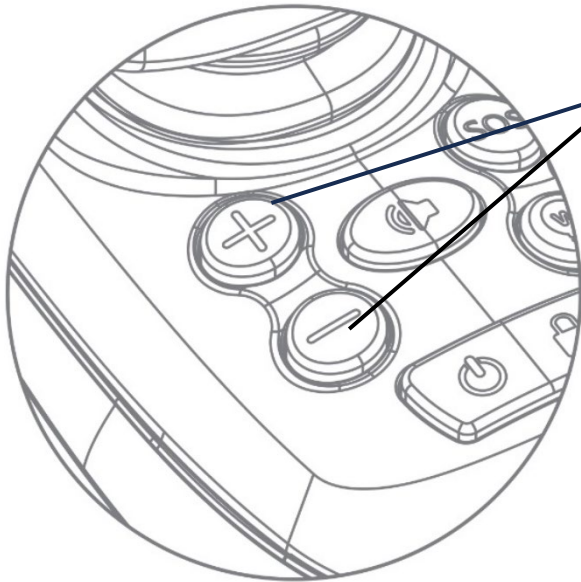


Enter pushing mode


3.6 Charging of Electric wheelchair.

Charging method, I: Plug the charger into the socket located under the control system firstly and then plug it into the power socket on the wall.

Charging method II: Remove the battery case and plug the charger into the charging socket on the case and then plug it into the power socket on the wall.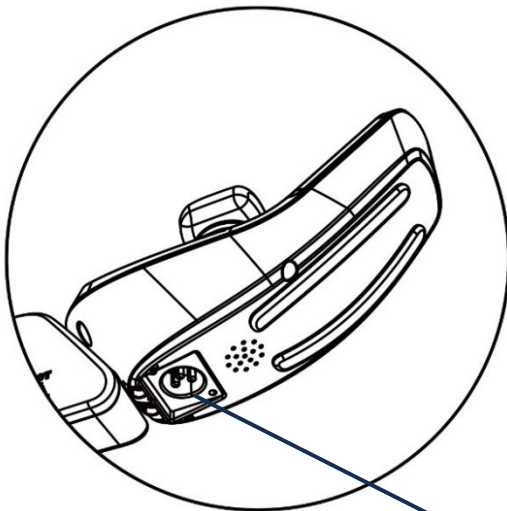

Charging socket under the control system.

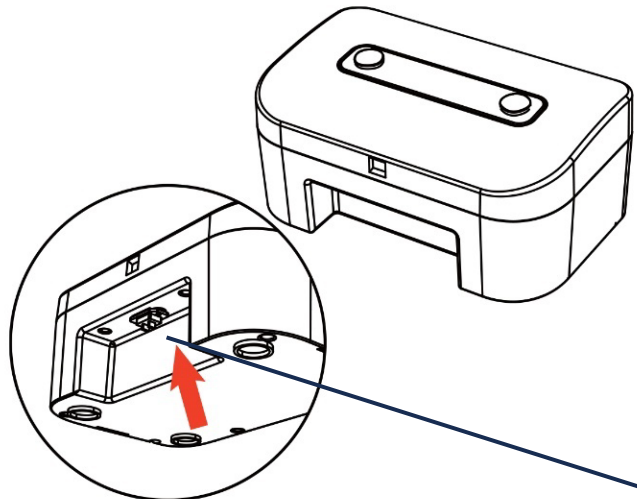Charging socket under the battery.

4. Maintenance and troubleshooting

4.1 Daily spot inspection and maintenance scheme

- Regularly (weekly or monthly, depending on the use frequency) check the connecting parts, such as screws and nuts, etc. of the frame for loosening, falling and corrosion, etc.
- Regularly check the movable joint parts for jam and falling, etc.
- Regularly check the tires for aging and serious wear, etc.
- Check the wheel pressure, If there are pneumatic wheels on the wheelchair.
- Clean the metal parts of the product with a dry cloth.
- During use of the Electric wheelchair, in case of rain or wet weather, the Electric wheelchair shall be wiped dry as soon as possible to prevent rust due to dampness.
- Exposure of the electric wheelchair to solar radiation under high temperature conditions should be avoided.
- We recommend a full review once a year to a qualified service provider.
- In case of repair, use only authentic spare parts.

4.2 Codes Fault Table

When contacting our technical support department at services@mobiak.com or our call center for a fault report or repair, please quote the error code.

Thanks to the built-in diagnostic control system of the device, the displayed error code may correspond to a problem during operation of the wheelchair.

Fault code	Situation	Action
E1	Key function abnormality	Fault alarm
E2	Joystick function abnormality	Fault alarm
E3	GPRS fault	Fault alarm
E4	Battery fault	Fault alarm
E5	Relay fuse fault	Fault alarm
E6	Equipment loss	Fault alarm
E7	Bluetooth fault	Fault alarm
E8	Motor fault	Fault alarm
E9	Driver fault	Fault alarm
EA	FLASH fault	Fault alarm

4.3 Safety warnings.

The device has the following audio voices.

Warning voice	Corresponding scenario
Enter novice mode	Press the key for novice mode
Quit novice mode	Press the key for novice mode again
Alarm	Keep a long press of SOS key
Lock	The default boot mode is lock mode, and the non lock to lock state transition
Unlock	Switch locking status to non-locking status
Wireless remote mode	Operate by the control key on the remote
Quit remote control mode	Operate by the control key on the remote
Fold	Keep a long press of horn key and SPD "-" key at the same time
Expand (Unfold)	Keep a long press of horn key and SPD "+" key at the same time
Upgrade mode	When the APP detects new version, click to upgrade
Upgrade finished	It will be triggered after single module is upgraded
Upgrading	The equipment is upgrading
Attention, reserve	Reserving, warning tone
Powering on tone	Power on, and the powering on tone will ring after 3 seconds
Charging	Charging
Charging completed	It will be triggered after charging is completed
Band-type brake open	The band-type brake of the equipment opens
Power off tone	Power off, and the powering off tone will ring after 3 seconds
Equipment bumpiness	It will be triggered for equipment bumpiness
Equipment slant	It will be triggered when the equipment is under slant,
Sharp bend	it will be triggered when the equipment is under sharp bend
Upgrade failed	There is something wrong during upgrading
Version V1.0	Voice version
Lower power, please charge	The battery life is lower than 20%, and it will be triggered in each minute
Stop Electric wheelchair folding.	Stop Electric wheelchair folding
Stop Electric wheelchair expanding	Stop Electric wheelchair expanding
Pushing mode	Press the "+" and "-" keys simultaneously
Quit pushing mode	Push the joystick to quit mode under pushing mode

4.4 Cleaning the product.

In addition to maintenance and inspection, a general cleaning for the frame and the upholstery can be carried out.

DO NOT leave the wheelchair wet or dirty with dust or mud. The frame should be cleaned regularly.

Use a clean and soft cloth to wipe the surface of the electric wheelchair.

6. Environmental Protection Statement:

If one day you find that your product needs to be replaced or it is no longer working for you, consider protecting the environment:

- 1) Do not dispose your product along with the rest of the public waste (this is also the meaning of the shown recycling sign).
- 2) Contact your Public Authorities and they will instruct you of the Recycling centers to which your product must be disposed.
- 3) Correct disposal of your product helps the protection of the environment as well as the recycling of the product's components.
- 4) The electrical products due to their construction materials, if not correctly disposed, may lead to environmental and furthermore health hazards

6. Incident Reporting:

Any serious incident that occurs in relation to use of the device must be reported to the manufacturer and a competent authority of the Member State in which the user and / or patient resides.

7. Declaration Of Conformity:

We are solely responsible for declaring that the Medical Devices mentioned in this statement are of Low-Risk Class (Class I) and comply with the requirements of the European Regulation 745/2017 and where appropriate, the standards and legislation referred to.

8. Terms of Warranty:

The product is guaranteed for a period of two (2) years warranty on the frame and six (6) months the battery, from the purchasing date.

The warranty apply on factory defects & DOES NOT cover damages resulting from misuse, poor maintenance, modification, overuse, or non-compliance with the product's operating instructions. Also DOES NOT cover the parts that can wear out during use or over time, like the upholstery, wheels, wheel tire, safety belt, footrests or brakes.

Damages or defects caused by: natural disasters, unauthorized maintenance or repair, power supply problems (where applicable) & carriers are not covered by the warranty.

No service or spare part is covered unless it has been inspected by an authorized service department or the manufacturer of the product. Shipping costs for goods of warranty are to be paid by the user-customer or by dealer.

When the user or the provider apply on warranty request, provide a valid proof of purchase and the completed warranty form, with a photo of the product serial number and a photo or video of the problem that has occurred. In the event the user cannot provide the above documents, if we can confirm that the product or part is still within the warranty period, we may be able to honor the user's request for coverage under warranty, but we are not obligated to do so.

Warranty Form

BUYERS DETAILS:			
FULL NAME:			
PHONE:			
PURCHASING DATE:		Invoice Number:	
SERIAL NUMBER:			
LOT:			
DEALER DETAILS:			
FULL NAME:			
ADDRESS:			
PHONE:			
			DEALER SIGN AND SIGNATURE

MOBIAK S.A
ΚΑΘΙΑΝΑ ΑΚΡΩΤΗΡΙΟΥ, ΧΑΝΙΑ - ΚΡΗΤΗ
KATHIANA AKROTIRIOU CHANIA - CRETE
T: +30 28210 63222 | WEB: www.mobiakcare.com

9. Disclaimer:

The content of the product manual is the intellectual property of MOBIAK S.A. (hereinafter referred to as the "Company") and the copyright is reserved by the Company. Without the written consent of the Company, no one may copy, export or translate the manual or any content included in any form into other languages without the written consent of the Company. The Company carefully compiles the product manual with a sense of responsibility for users but does not guarantee the completeness or correctness of its contents. The product manual, as a purely technical document, does not include implied content or content with oblique reference to third parties. The Company assumes no responsibility for any typographical errors that may cause ambiguities. In the event of direct or indirect loss of information or business interruption due to the product manual or the product information contained therein, the Company shall not be liable.

When using all MOBIAK S.A. products, users must strictly follow the product manual and the instructions for use available on the Company's official website (www.mobiakcare.com) for the handling of the equipment and not violate any warning marked in the product manual when using the equipment. In case of injury to users or other persons, or material damage caused by the handling of the equipment without following the instructions for use or by violating the warnings marked in the product manual, the persons concerned shall bear the respective legal responsibilities.

As the Company continues to improve and update its products, it is essential that you regularly log on to its official website to refer to the most up-to-date instructions and conditions of use or contact your distributor for the latest user manual. The Company reserves the right to revise the product manual without notice.

The Company reserves the right of final interpretation.

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