MA ST34x\_ST35x\_ST36x 01 B\_05-2016

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I Class Medical Device  
according to Directive 93/42/EEC and further modifications

### 1. Introduction

Thank you for purchasing a SINFONIA cushion and/or pillow from Kyara line by Moretti. The SINFONIA cushions and pillows in memory foam are designed and realized to meet all your needs for a practical, proper and safe use. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

### 2. Intended use

**SWIVEL CUSHION:** useful device to help people with difficulty in moving the legs to get out of bed, to get up from chairs or get out from the car.

**ROUND SHAPE TRAVEL CUSHION:** useful device to provide support and relief to the neck area

**SEATING WEDGE CUSHION:** device used to keep the spine in balance and in its "S" natural form.

**LUMBAR SUPPORT:** useful device for giving support to the back and help to maintain proper posture

**SEMI-ROLL/ROLL CUSHION:** device useful as a support for the lower back, the neck area, legs and heels.

**PILLOW:** useful device to support the head and allow the spine to assume the correct position during sleep, warranty for a perfect rest.

#### CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

### 3. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**

4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

#### **4. General warnings**

- READ carefully the entire manual before using the device
- Make sure that the product is delivered complete and keep it packed away from heat sources
- SERVICE LIFE- the device use limit is defined by the wear of the parts
- DO NOT allow children to play or operate the device

#### **5. General description**

##### **5.1 Cushions codes**

###### **ST340** Swivel Cushion

Swivel cushion designed to help people with difficulty in moving his legs to get out from his bed, to get up from the chair. Made with non-slip, padded seat and pivoting on a rigid procedure in order to reduce the effort for the user.

###### **ST343** Round shape travel cushion

The cervical collar in memory foam, thanks to its round shape provides a firm but comfortable support. Ideal in the presence of cervical disease neurological attorney relief to muscle tension in the neck area, ideal when traveling or reading while seated or in bed. Completely removable and washable polyester cover.

###### **ST345** Seating wedge cushion

The seating wedge cushion in memory foam allows you to keep your spine in balance and in its "S" natural shape. The wedge shape pushes the pelvis forward favoring the uniform distribution of the weight over the entire surface of the seat and helping to maintain the natural curve of the column so as to obtain a sitting position healthier and more comfortable as to help to reduce fatigue and stress.

###### **ST346** Lumbar support

Lumbar support ideal for travel or work. Made of molded polyurethane and uncut. It is used between the back of the seat and lumbar back to give support to the back and help maintain proper posture. Useful to reduce back pain when sitting or driving for a long time. Complete with adjustable straps.

###### **ST350S** Semi roll cushion - Small

###### **ST350L** Semi roll cushion - Large

The SEMI ROLL cushion is a multi-purpose memory foam cushion. Due

to its shape and the availability in two lengths, can be used behind your back, as a support for the lower back, under your head to support the neck area, under your knees to relieve the legs while you are sitting or lying down in bed or under the heels to prevent heel pain and sores. Complete with washable polyester.

### **ST352** Cylindrical cushion Twist

The cylindrical cushion twist ensures focused support to the painful area. When folded and stored it remains in that position for a complete support. Thanks to its anatomical shape and well articulated, the cushion is designed to fit around any body part where a support is appropriate. Wrapped around the neck and/or on side of your head to relieve the stress or around the lower back or over your shoulder it gives a perfect support and to relieve muscle fatigue. It is compact, easy to carry and comfortable while reading, resting, sleeping, watching TV, travelling, and so on...

## **5.2 Pillows codes**

### **ST365 - ST360 - ST362 - ST363**

The SINFONIA memory foam pillows from Kyara line by Moretti, offer an amazing variety of possibilities for those people looking for the perfect pillow, can preserve from annoying discomfort and ensure a safe and peaceful rest. The memory foam is a gel-foam able to "memorize" the shape of the head and neck as it is sensitive to body heat. The memory foam pillows create the perfect support and distribute pressure evenly so as to ensure proper spinal posture during sleep by relieving pressure on the neck and offering relief to the shoulders. Indicated in case of headache, neck pain, shoulder pain and insomnia. Complete with polyester lining completely removable.

There are different models available, shaped models (**ST360-362**) and classical model (**ST365**). They are hypoallergenic, well breathable, ideal for those looking for ergonomics, practicality and attention to health.

Thanks to the PU Memory Foam pillow (**ST363**) you can comfortably sleep on your back or side with every kind of mask as its anatomical shape. The soft Memory Foam stuffing and its natural cotton cover let you to lay your head on a soft material naturally adaptable to its shape. This pillow for NIV, CPAP, APAP prevents mask leaks due to its support on it. The pillow supports your head, neck and shoulders while providing a correct positioning of the respiratory tract. It helps to reduce the possibility of postural apneas as well.

The aim of the CPAP pillow is to increase the efficiency of the C-PAP machine as facilitate a better sleep comfort.

## **6. Before use**

Always check the state of wear of the parts in such a way as to ensure

it is used in complete safety for persons and property.

### 7. Warnings for use

- Avoid to leave the device in contact with water for long periods of time
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks.

### 8. Maintenance

The KYARA devices by Moretti are checked carefully and supplied with EC mark, once launched on the market.

### 9. Cleaning

Washing indication for cover, where present:

- Maximum temperature: 30°C
- Hand wash, use neutral and delicate products
- Do not bleach
- Do not iron (do not use steam)
- Do not dry clean, do not use stain removers with solvents
- Do not tumble dry

**Note:** The foam can not be washed

### 10. General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

### 11. Technical features

CODES	MATERIAL	COVER	SIZE
ST340	100% Polyester Non-sliding base	100 % cotton	44x44x4 cm
ST343	100% Polyurethane	100% polyester	30x35x9 cm
ST345	100% Polyurethane	100% polyester	41x41x3/7.5 cm
ST346	100% Polyurethane	100% polyester	33x35x9/3 cm
ST350S	100% Polyurethane	100% polyester	35x20x10 cm
ST350L	100% Polyurethane	100% polyester	50x20x10 cm
ST352	100% Polyurethane Core in PU	100% polyester	65xR12 cm
ST360	100% Polyurethane	100% polyester	60x30x9/11 cm
ST362	100% Polyurethane	100% polyester	25x30x9/11 cm
ST363	100% Polyurethane	100 % cotton	55x37x7/12 cm
ST365	100% Polyurethane	100% polyester	66x45x13 cm

## 12. Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below.

The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed.

The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual. The warranty doesn't cover components subject to wear and tear during the correct use of the device.

### 12.1 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference.

Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free.

The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE**

Product \_\_\_\_\_

Purchased on (date) \_\_\_\_\_

Retailer \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

Sold to \_\_\_\_\_

Address \_\_\_\_\_ Town/city \_\_\_\_\_

**MORETTI S.P.A.**

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

[www.morettispa.com](http://www.morettispa.com) email: [info@morettispa.com](mailto:info@morettispa.com)**MADE IN P.R.C**

Via Bruxelles, 3 - Meleto  
52022 Cavriglia (Arezzo)  
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200  
[www.morettispa.com](http://www.morettispa.com)  
[info@morettispa.com](mailto:info@morettispa.com)